
POLSKI

ENGLISH

REH4MAT
36-060 Głogów Małopolski, ul. Piaski 47
tel. +48 (016) 621 41 35, fax: +48 (016) 621 42 13
e-mail: biuro@reh4mat.com, www.reh4mat.com, www.probraces.com
The company holds the ISO 13485 international
quality management system certificate
DATA AKTYWALIZACJI / THE UPDATE DATE OF THE
INSTRUCTION: 01.07.2015

REH4MAT
MANUFACTURER OF MODERN ORTHOPAEDIC BRACES

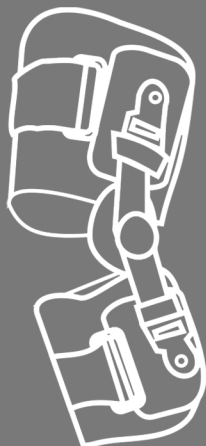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KNEE BRACE

ATTENTION: BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTION COMPLETELY AND CAREFULLY AND CHECK IF BRACE'S ELEMENTS AND SEAMS ARE SEEDY. ONLY UNDAMAGED DEVICE AND ITS CORRECT APPLICATION ARE VITAL TO THE PROPER FUNCTIONING OF IT. SAVE THE PRODUCT INFORMATION, TO BE ABLE TO REREAD THE INSTRUCTION LATER.

PRODUCT DESCRIPTION:

The knee brace is an excellent solution for effective knee therapy in case of injury or other conditions. Braces are appreciated by specialists (physicians, physical therapist) and patients. Reh4Mat products are innovative and are carefully manufactured of the high quality materials. Our key objective is customer's satisfaction. In case of any questions feel free to contact with us by e-mail: biuro@reh4mat.com

IMPORTANT:

- Proper use of the brace requires diagnosis by physician or physical therapist.
- Optimal brace functioning can only be achieved by selecting the right size. The Reh4Mat brace must be fitted by an authorized vender, physician or educated psycho-therapist.
- It is necessary to control using the brace, especially in children, people with memory dysfunction, psychiatric and people with sensory disturbances.
- The skin in place, where the brace touches should be clean (the creams or ointments should not be used).
- Is it necessary to control the skin's condition and in case of skin lesions occurring is it necessary to contact with physician.
- In case of skin lesions (epidermis scrapes, scatches or eczema) using of the braces is allowed after dressing applying.
- Chronic using the brace may cause limitation of the active and passive movement in joints. If there are no contraindications, it is recommended to exercise several times during the day. Exercises are selected individually for each patient by a master's degree or technician rehabilitation as physician has recommended.
- In case of varicose veins, Deep Vein Thrombosis (DVT), lower limb oedema the using of brace is allowed only after contact with physician.
- Use of the brace during sleeping should be done only if this was prescribed by your doctor.

CARE AND MAINTENANCE:

- Wash by hands in warm water with soap in temperature of 30 C,
- Sluice carefully,
- Do not iron,
- Do not chlorinate,
- Do not dry clean,
- Do not tumble dry,
- Protect from moisture,
- Protect from high temperatures,
- Dry away from sources of warmth.

CAUTION: IF THE DEVICE HOLDS LATERAL SPLINTS, SHELLS AND IT IS POSSIBLE TO DISMANTLE THEM WITHOUT ANY TOOLS, BEFORE WASHING REMOVE THE SPLINTS.

ATTENTION: Pain is always the sign of warning. If the pain maintains or increases it is advisable to consult your doctor or physical therapist.
ATTENTION: FOR SINGLE PATIENT USE ONLY. USING OF THE DEVICE NOT ACCORDING TO THE MANUAL MAY CAUSE SERIOUS COMPLICATIONS.