

KĄT ZGIĘCIA LUB WYPROSTU USTALA LEKARZ PROWADZĄCY LUB REHABILITANT

1. Ustawić ortezę w przybliżonym położeniu, w którym chcemy ograniczyć zgięcie (wyprost) rys.1.
2. Wkręcić wkręt (wkręty) w odpowiednie miejsca oznaczone na szynie rys.2.

- A- Ograniczenie kąta wyprost w np. 30°, rys.3
- B- Ograniczenie kąta zgięcia w np. 60°, rys.3.

UWAGA:

PRZED USTAWIENIEM KĄTA ZGIĘCIA LUB WYPROSTU, ZALECA SIĘ ZAŁOŻYĆ WYRÓB NA KOŃCZYNĘ
Jeżeli orteza wyposażona jest w dwie szyny boczne, regulacji należy dokonać osobno na obydwu szynach bocznych, ustawiając ten sam kąt zgięcia lub wyprost (punkt 1-2).
Szyny nie wymagają konserwacji.


Termin ważności: NIE OKREŚLA SIĘ ☞ 2013
Okres gwarancji: 24 MIESIĘCE OD DATY ZAKUPU

Konsultant Med: tel. 508 382 509



Podstawa dopuszczenia do obrotu:
Produkt został zarejestrowany w Urzędzie Rejestracji Produktów
Leczniczych, Wyrobów Medycznych i Produktów Biobójczych w
Wyrób klasy I

REH4MAT Sławomir Wroński
ul. Piaski 47, 36-060 Głogów Młp., Polska
tel. +48 16 621 42 20; fax +48 16 621 42 13
e-mail: biuro@reh4mat.com, www.reh4mat.com

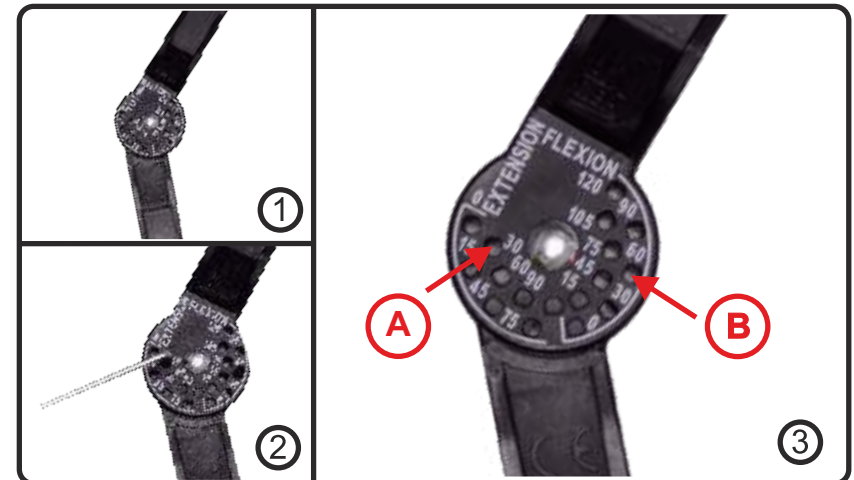

THE ANGLE OF BENDING AND STRAIGHTENING IS SPECIFIED BY THE DOCTOR OR PHYSIOTHERAPIST

1. Place the support in the position in which you want to stabilize the flection (extension) (1).
2. Place the screws in the holes (2).

- A- restriction of the extension angle at e.g. 30 degrees (3)
- B- restriction of the flection angle at e.g. 60 degrees (3)

ATTENTION:

first put the device on the limb than adjust the angle of bending and straightening.

If the orthosis is equipped with 2 side splints, adjustment is done on each side splint separately, setting the same angle of bending and straightening on both splints (point 1-3).

Side splints do not require maintenance.

Expiry date - IS NOT DETERMINED. ☞ 2013

Guarantee period - 24 months from the date of purchasing.



Admitted on the market: product registered
in the Department of Registration of Healing Products,
Medical Products and Biocide Products.
Medical product first class

REH4MAT Sławomir Wroński
ul. Piaski 47, 36-060 Głogów Młp., Poland
tel. +48 16 621 42 20; fax +48 16 621 42 13
e-mail: biuro@reh4mat.com, www.reh4mat.com