


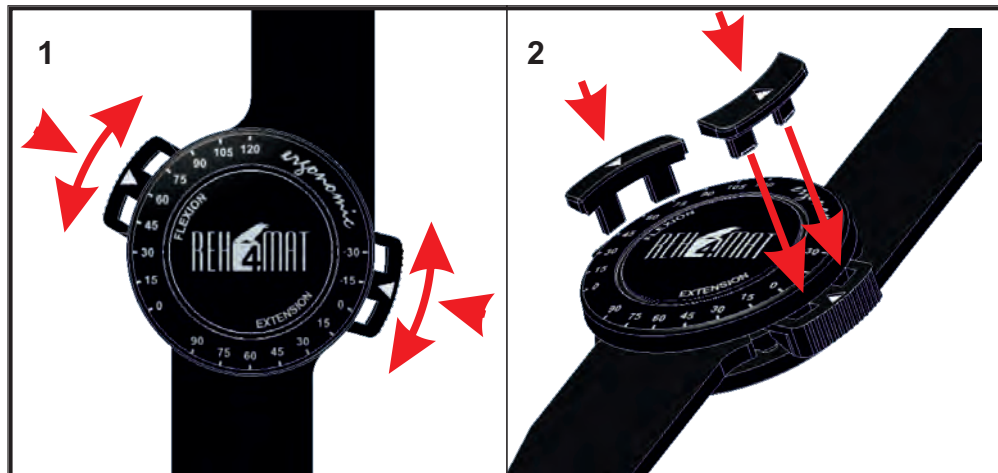

НЕОБХОДИМЫЙ УГОЛ СГИБАНИЯ / РАЗГИБАНИЯ ОПРЕДЕЛЯЕТ ВРАЧ ИЛИ ФИЗИОТЕРАПЕВТ

Боковые шины этого ортеза имеют регулируемые ограничители угла сгибания в 0°, 15°, 30°, 45°, 60°, 75°, 90°, 105°, 120°, разгибания 90°, 75°, 60°, 45°, 30°, 15°, 0°, и перегибания -15°, -30°.

1. Согнуть ортез под желаемым углом ограничения сгибания/разгибания.
2. Прижимая ограничители (1) к центру шарнира переместить их по ободку в нужное положение с обозначенным углом.
3. Зафиксировать ограничители специальными фиксаторами (2) поставляемыми в комплекте с шиной.

ВНИМАНИЕ:

ПЕРЕД ФИКСАЦИЕЙ УГЛА СГИБАНИЯ/РАЗГИБАНИЯ СЛЕДУЕТ НАЛОЖИТЬ ИЗДЕЛИЕ НА КОНЕЧНОСТЬ.
Если на изделии 2 боковые шины, необходимо установить одинаковый угол на каждой шине.

Боковые шины не требуют обслуживания.


Срок годности - БЕЗ СРОКА ГОДНОСТИ. ☞ 2013

Гарантийный срок: 24 МЕСЯЦА С ДАТЫ ПОКУПКИ



Допуск на рынок: продукт зарегистрирован
в Управлении по регистрации медикаментов,
Медицинских изделий и биопрепаратов.
Изделие класса I

REH4MAT Sławomir Wroński
ul. Piaski 47, 36-060 Głogów Młp., Poland
tel. +48 16 621 42 20; fax +48 16 621 42 13
e-mail: biuro@reh4mat.com, www.reh4mat.com

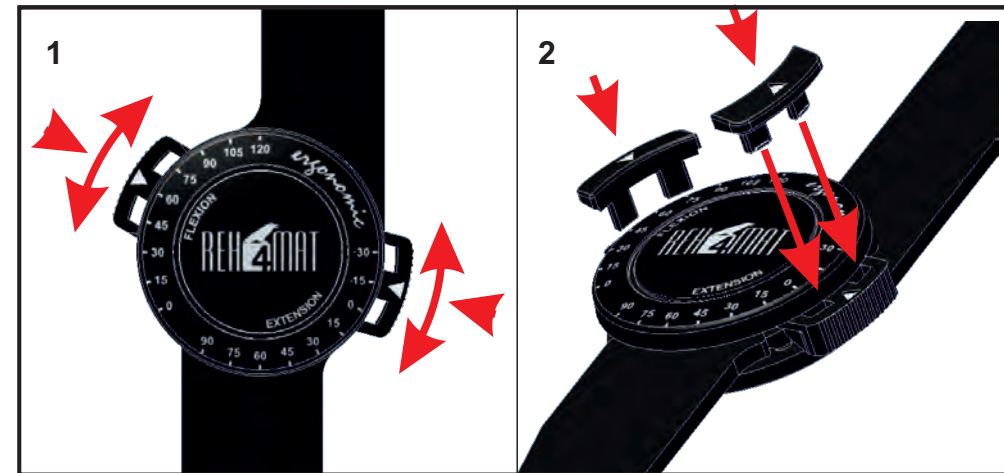

THE ANGLE OF BENDING AND STRAIGHTENING IS SPECIFIED BY THE DOCTOR OR PHYSIOTHERAPIST

FLEXION: 0°, 15°, 30°, 45°, 60°, 75°, 90°, 105°, 120°. EXTENTION: 90°, 75°, 60°, 45°, 30°, 15°, 0°, -15°, -30°.

1. Place the support in the position in which you want to stabilize the flexion (extension).
2. By pressing the buttons - delimiters (1) to the center of the splint move them around to the desired place with the prescribed angle.
3. Lock buttons with the special lockers (2) that came with splint.

ATTENTION:

First put the device on the limb than adjust the angle of bending and straightening.

If the orthosis is equipped with 2 side splints, adjustment is done on each side splint separately, setting the same angle of bending and straightening on both splints.

Side splints do not require maintenance.

Expiry date - IS NOT DETERMINED. ☞ 2013

Guarantee period - 24 months from the date of purchasing.



Admitted on the market: product registered
in the Department of Registration of Healing Products,
Medical Products and Biocidal Products.
Medical product first class

REH4MAT Sławomir Wroński
ul. Piaski 47, 36-060 Głogów Młp., Poland
tel. +48 16 621 42 20; fax +48 16 621 42 13
e-mail: biuro@reh4mat.com, www.reh4mat.com