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KNEE LIGAMENT BRACE WITH ANATOMIC ROM ADJUST-MENT ATOM 2RA

ATTENTION: BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTION COMPLETELY AND CAREFULLY AND CHECK IF BRACE'S ELEMENTS AND SEAMS ARE SEEDY. ONLY UNDAMAGED DEVICE AND ITS CORRECT APPLICATION ARE VITAL TO THE PROPER FUNCTIONING OF IT. SAVE THE PRODUCT INFORMATION, TO BE ABLE TO REREAD THE INSTRUCTION LATER.

PRODUCT DESCRIPTION:

The knee brace is an excellent solution for effective knee therapy in case of injury or other conditions. Braces are appreciated by specialists (physicians, physical therapist) and patients.

Reh4Mat products are innovative and are carefully manufactured of the high quality materials. Our key objective is customer's satisfaction.

In case of any questions feel free to contact with us by e-mail: biuro@reh4mat.com

PURPOSE OF USE:

- injuries,
- degenerative conditions,
- pain syndrome,
- knee ligaments injuries (ACL),
- conditions after ligament reconstruction (ACL).

IMPORTANT:

- Proper use of the knee brace requires diagnosis by physician or physical therapist.
- Optimal brace functioning can only be achieved by selecting the right size. The Reh4Mat knee brace must be fitted by an authorized vender, physician or educated psychotherapist.
- It is necessary to control using the brace, especially in children, people with memory dysfunction, psychiatric and people with sensory disturbances.
- The skin in place, where the brace touches should be clean (the creams or ointments should not be used).
- Is it necessary to control the skin's condition (especially in heels and feet) and in case of skin lesions occurring is it necessary to contact with physician.
- In case of skin lesions (epidermis scrapes, scatches or eczema) using of the braces is allowed after dressing applying.
- Chronic using the brace may cause limitation of the active and passive movement in joints. If there are no contraindications, it is recommended to exercise several times during the day. Exercises are selected individually for each patient by a master's degree or technician rehabilitation as physician has recommended.
- In case of varicose veins, Deep Vein Thrombosis (DVT), lower limb oedema the using

of brace is allowed only after contact with physician.

- Use of the brace during the night should be done only if this was prescribed by your doctor.

CARE AND MAINTENANCE:

- Wash by hands in warm water with soap in temperature of 30 C,
- Sluice carefully,
- Do not iron,
- Do not chlorinate,
- Do not dry clean,
- Do not tumble dry.
- Protect from moisture,
- Protect from high temperatures,
- Dry away from sources of warmth.

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Symbol	ATOM 2RA
Trade name	SUPPORT, LOWER-EXTREMITY
Model	SUPPORT, LOWER-EXTREMITY AM-OSK-ZL/3

ATTENTION: Pain is always the sign of warning. It the pain maintains or increases it is advisable to consult your doctor or physical therapist. ATTENTION: FOR SINGLE PATIENT USE ONLY. USING OF THE DEVICE NOT ACCORDING TO THE MANUAL MAY CAUSE SERIOUS COMPLICATIONS.

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