POLSKI

ENGLISH

36-060 Głogów Małopolski, ul. Piaski 47
 ul. Truskawkowa 17, Widna Góra, 37-500 Jarosław, POLAND
 tel. +48 (016) 621 41 35, fax: +48 (016) 621 42 13
 Mailing address/Adres mailowy:
 biuro@reh4mat.com
 www.reh4mat.com; www.probraces.com
 The company holds ISO 13485 international
 quality management system certificate
 DATA AKTUALIZACJI / THE UPDATE DATE OF THE
 INSTRUCTION: 24.09.2015

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BACK BRACE

ATTENTION: BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTION COMPLETELY AND CAREFULLY AND CHECK IF BRACE'S ELEMENTS AND SEAMS ARE SEEDY. ONLY UNDAMAGED DEVICE AND ITS CORRECT APPLICATION ARE VITAL TO THE PROPER FUNCTIONING OF IT. SAVE THE PRODUCT INFORMATION, TO BE ABLE TO REREAD THE INSTRUCTION LATER.

PRODUCT DESCRIPTION:

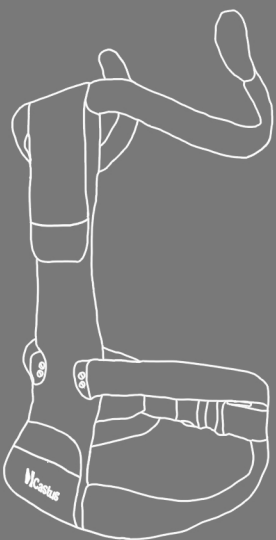
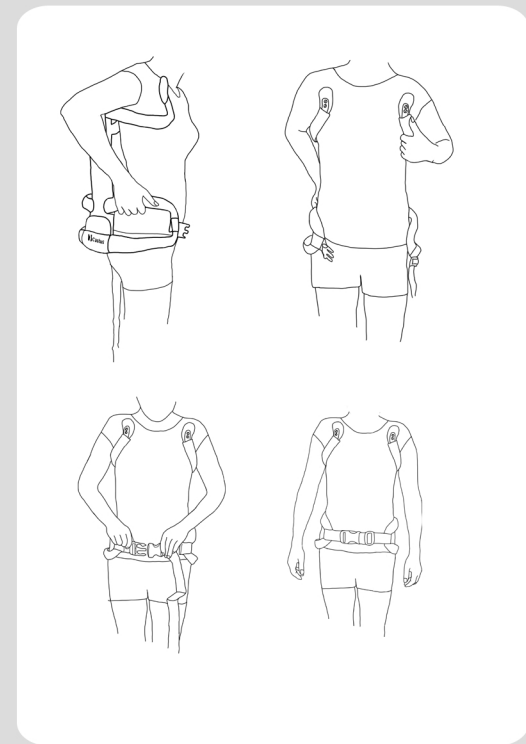
Back brace by stabilization provides support in the spine during daily actions and decreases pain. Reh4Mat products are innovative and are carefully manufactured of the high quality materials. Our key objective is customer's satisfaction. In case of any questions feel free to contact with us by e-mail: biuro@reh4mat.com

IMPORTANT:

- Proper use of the brace requires diagnosis by physician or physical therapist.
- Optimal brace functioning can only be achieved by selecting the right size. The Reh4Mat brace must be fitted by an authorized vender, physician or educated physio-therapist.
- The brace should be inserted on a cotton underwear, to avoid skin abrasions and skin sores.
- It is necessary to control using the brace, especially in children, people with memory dysfunction and psychiatric.
- The skin in place, where the brace touches should be clean (the creams or ointments should not be used).
- Is it necessary to control the skin's condition and in case of skin lesions occurring is it necessary to contact with physician.
- In case of skin lesions (epidermis scrapes, scatches or eczema) using of the brace is allowed after dressing applying.
- Chronic using the brace may cause limitation of the active and passive movement in joints. If there are no contraindications, it is recommended to exercise several times during the day. Exercises are selected individually for each patient by a master's degree or technician rehabilitation as physician has recommended.
- In case of Deep Vein Thrombosis (DVT), oedema the using of brace is allowed only after contact with physician.
- Use of the brace during the night should be done only if this was prescribed by your physician.
- In case of urinary incontinence and acute urinary tract infections the use of brace should be consulted with physician.
- In case of symptoms from the liver and gallbladder the use of brace should be consulted with physician.

CARE AND MAINTENANCE:

- Wash by hands in warm water with soap in temperature of 30 C,
 - Sluice carefully,
 - Do not iron,
 - Do not chlorinate,
 - Do not dry clean,
 - Do not tumble dry,
 - Protect from moisture,
 - Protect from high temperatures,
 - Dry away from sources of warmth.
- ATTENTION: IF THE DEVICE HOLDS FRAME AND IT IS POSSIBLE TO DISMANTLE IT WITHOUT ANY TOOLS, BEFORE WASHING REMOVE THE FRAME.**

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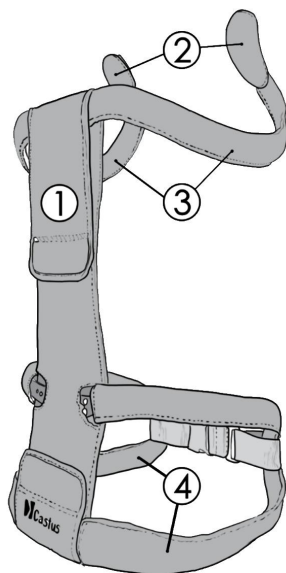
ATTENTION: Pain is always the sign of warning. If the pain maintains or increases it is advisable to consult your doctor or physical therapist.
ATTENTION: FOR SINGLE PATIENT USE ONLY. USING OF THE DEVICE NOT ACCORDING TO THE MANUAL MAY CAUSE SERIOUS COMPLICATIONS.

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
ATTENTION: Adjustment of the product shall not be done on the user.
To adjustment use screwdriver attached to the product.

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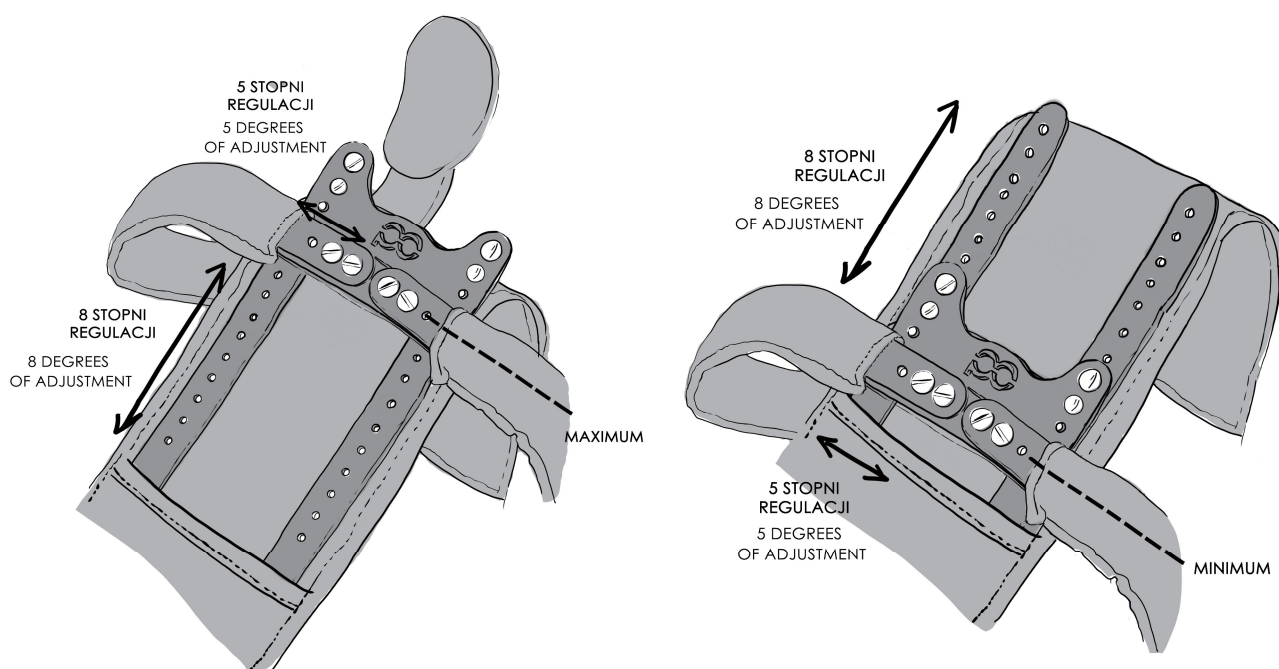
Unbuckle the upper part [1] to adjust the frame by screwdriver. You can adjust width and height of the product. There are two variants of settings:

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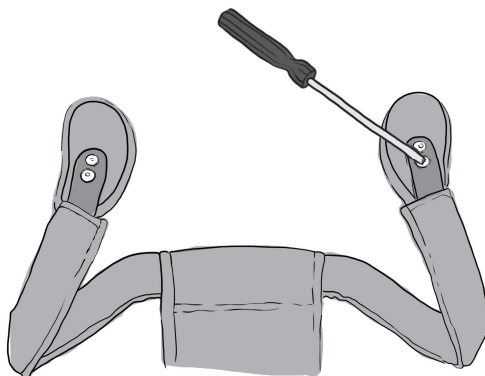
Use a screwdriver to adjust the height of the front parts [2]

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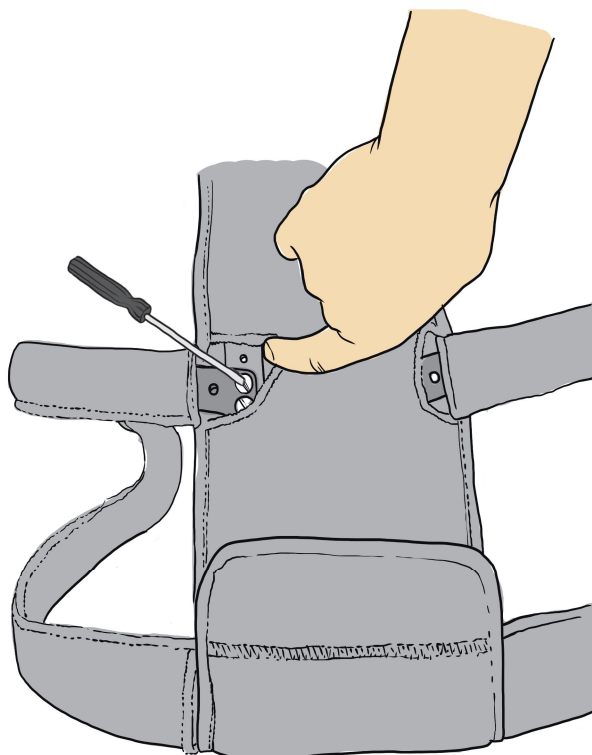Adjust the hip frame [4] adequately to the user.

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If required, there is the possibility of bending the top frame [3], whole frame and hip frame [4] adequately to the user.