

KĄT ZGIĘCIA LUB WYPROSTU USTALA LEKARZ PROWADZĄCY LUB REHABILITANT

1. Ustawić ortezę w przybliżonym położeniu, w którym chcemy ograniczyć zgięcie (wyprost)
2. Wkręcić wkręt (wkręty) w odpowiednie miejsca oznaczone na


A- Ograniczenie kąta wyprost w np. 30°, rys.3

B- Ograniczenie kąta zgięcia w np. 60°, rys.3

UWAGA:

Jeżeli orteza wyposażona jest w dwie szyny boczne, regulacji należy dokonać osobno na obydwu szynach, utawiając ten sam kąt zgięcia lub wyprost.

Szyny nie wymagają konserwacji

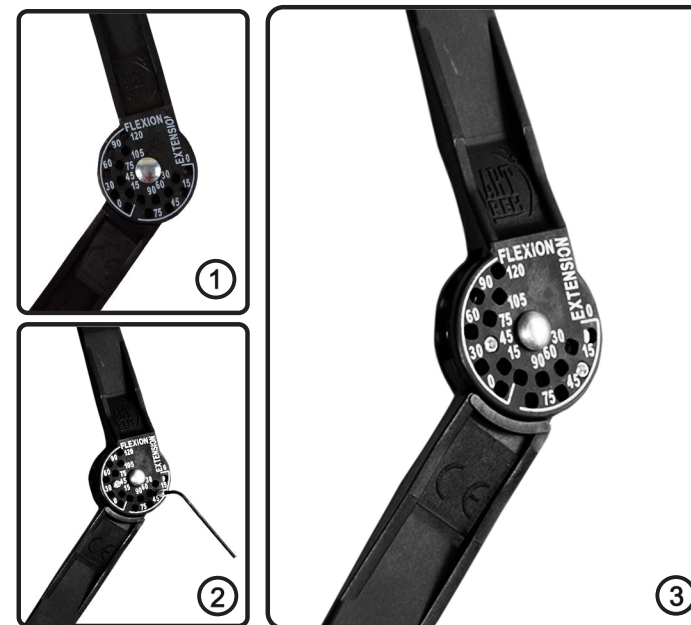

THE ANGLE OF FLEXION AND EXTENSION IS SPECIFIED BY THE DOCTOR OR PHYSIOTHERAPIST

1. Place the brace in the position in which you want to adjust the flexion/extension
2. Place the screws in the holes showed in the picture on the left.

A- restriction of the flexion angle e.g. 30° degrees (3)

B- restriction of the extension angle e.g. 60° degrees (3)

ATTENTION:

If the brace is equipped with 2 slide splints, adjustment is done on each side splint separately, setting the same angle of flexion and extension. Side splints do not require maintenance.