


KĄT ZGIĘCIA LUB WYPROSTU USTALA LEKARZ PROWADZĄCY LUB REHABILITANT

1. Podnieść (otworzyć) klapkę zabezpieczającą (pokrywę) podważając w miejscu zaczepu;
2. Nastawić zalecany kąt zgięcia lub wyprost;
3. Zablokować ramiona szyny (dwustronnie) wkładając we właściwe otwory 4 nity blokujące;
4. Zamknąć klapkę (pokrywę) zatrzaskując zaczep.

UWAGA:

Jeżeli orteza wyposażona jest w dwie szyny boczne, regulacji należy dokonać osobno na obydwu szynach bocznych, ustawiając ten sam kąt zgięcia lub wyprost (punkt 1-4). Szyny boczne nie wymagają konserwacji.

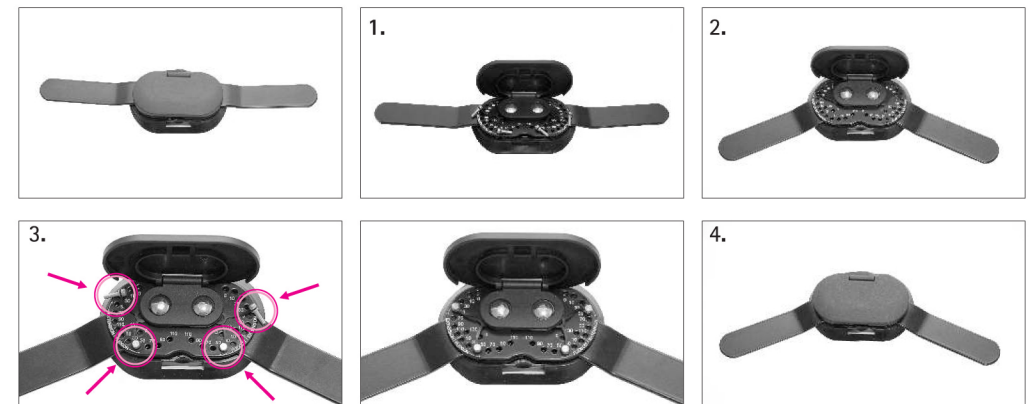

THE ANGLE OF FLEXION AND EXTENSION IS SPECIFIED BY THE DOCTOR OR PHYSIOTHERAPIST

1. Open the securing cup by lifting it in the place of attachment;
2. Adjust required flexion/extension position;
3. Lock the splint (on both sides) inserting into correct holes 4 locking rivets;
4. Close the cup (clicking sound).

ATTENTION:

If the brace is equipped with 2 slide splints, adjustment is done on each side splint separately, setting the same angle of flexion and extension on both splints (point 1-4). Side splints do not require maintenance.