BodyMapTM

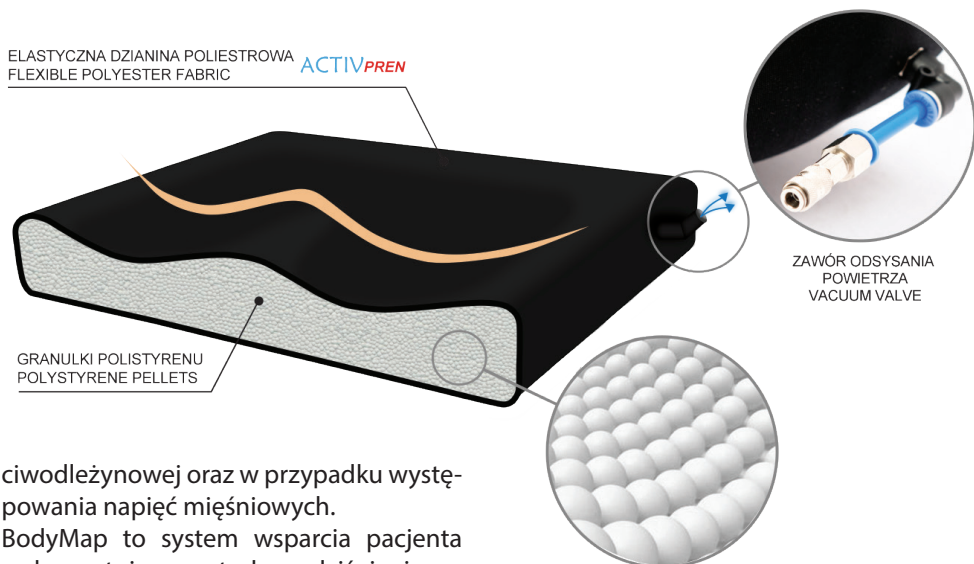
VACUUM SYSTEM BY ORTHODESIGN

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BodyMap System refers to first class medical products which are designed for adults as well as children with cerebral palsy. All these products provide perfect stabilization during seating or lying position. What is more, it is a perfect choice for the disabled due to the fact that it prevents bed sores.

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
BodyMap System uses vacuum method. This method refers to products which are filled with polystyrene granules thanks to which products can model according to patient's body shape. This method is well-known within The Western European countries and it is widely applied in stabilization process. Furthermore, it is also applied during transportation of patients in ambulances. Polystyrene granules move freely inside the device and, more to the point, it is easy to move them in order to shape cushion at your discretion. The shape of cushion doesn't change easily because of the fact that the air is sucked out of the cushion.

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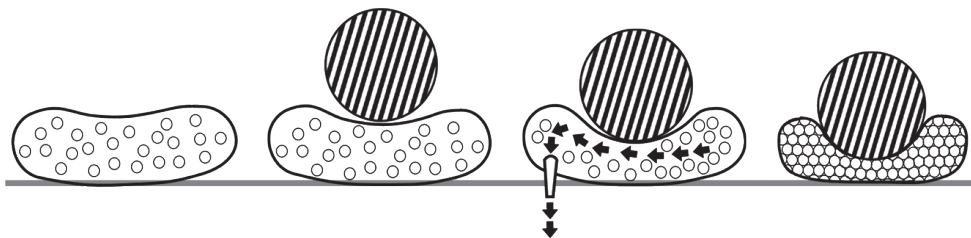Our system is designed with the intention of maintaining correct sitting. cushions, backrests and pads make it possible to have correct support for trunk, hips, head and upper as well as lower limbs. BodyMap System is perfect for patients with cerebral palsy, scoliosis or hump. The product is shaped according to patient's

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
body shape thanks to which unit pressure is diminished. It has a significant influence on bed sores prophylaxis. In addition, it is possible to determine the level of product's hardness with the aid of vacuum method.

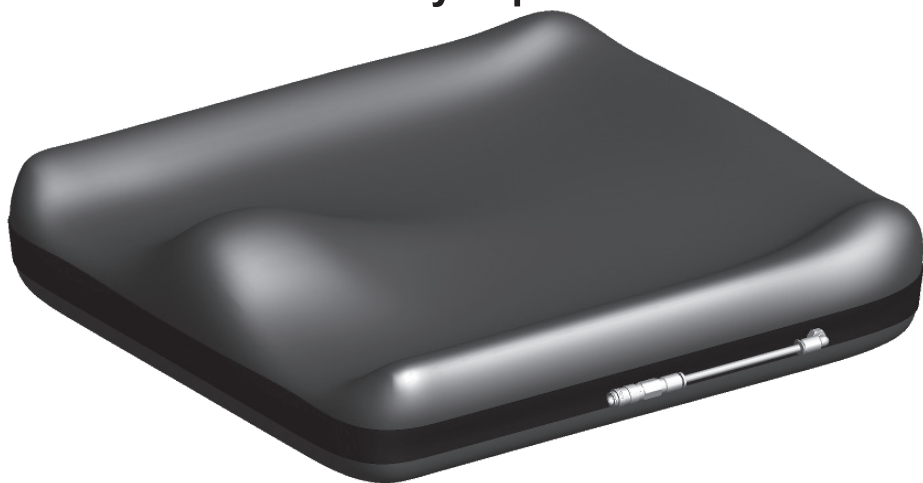BodyMap products are made of soft, elastic neoprene foam and they perfectly adjust to patient's body. cushions are laminated by means of elastic polyester knitwear and some of them can be additionally laminated with the aid of knitwear with self-adhesive qualities which makes it possible to use additional stabilizing tapes.

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
Concerning shaping the cushion, it is highly important to open valve so as to let the air in. This activity is done in order to equalize pressure inside the device with atmospheric pressure.

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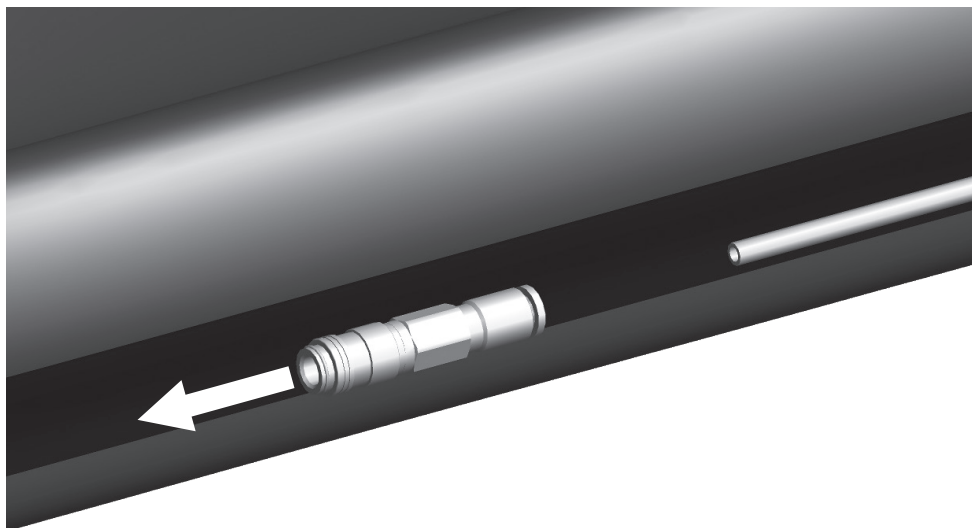
In order to open valve it is necessary to put thumb on frontal valve surface. Moreover, use your middle as well as index finger to press blue ring (1) and simultaneously slip off valve from pipe (2).

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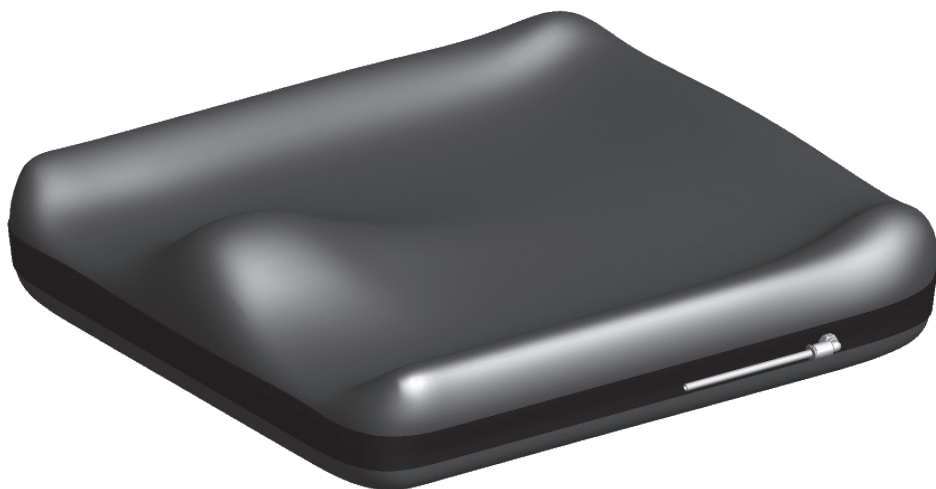
Be careful with the valve because it is small and it can be easily lost. Be cautious especially when you perform all activities outside.

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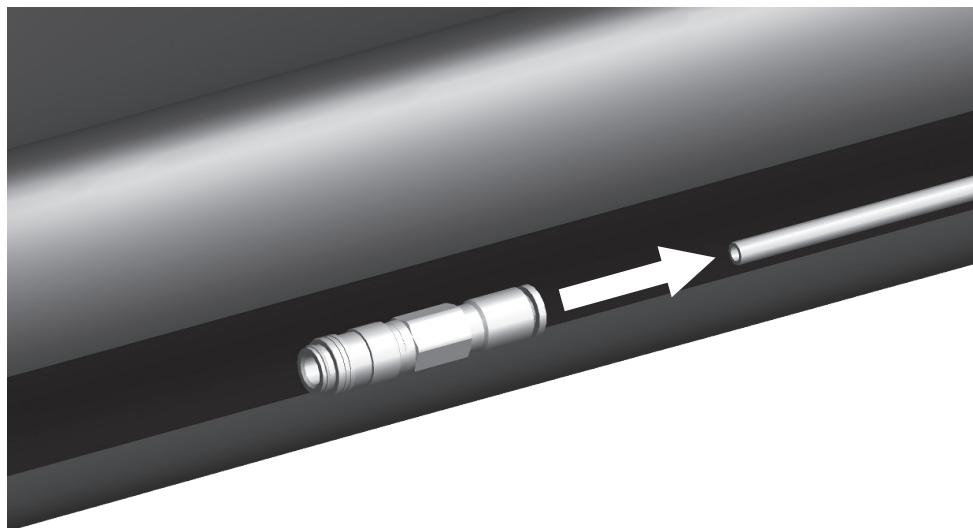
Cushion without valve should be left for few minutes only so as to equalize pressures.

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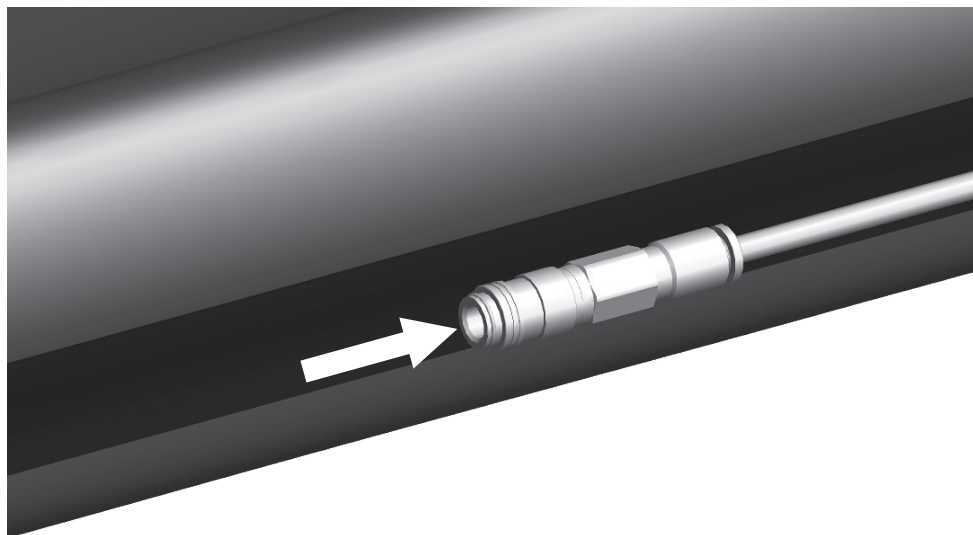
Insert valve on pipe so as to mount valve again.

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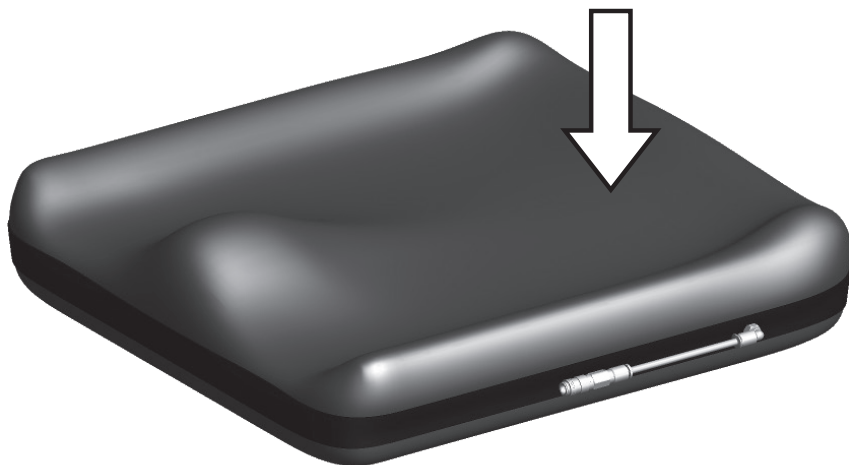
Valve is equipped with self-closing function and therefore valve doesn't have to be additionally mounted.

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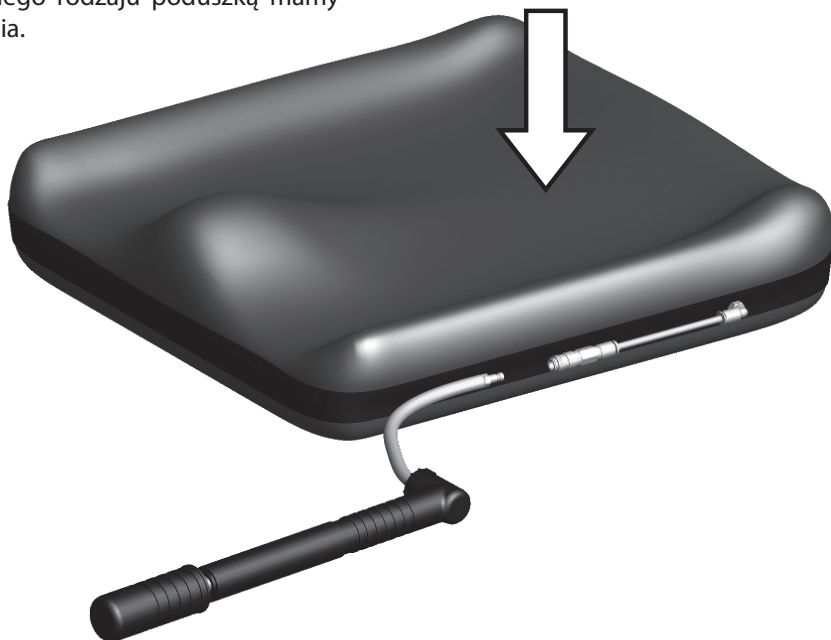
Next, shape cushion at your discretion.
You can shape it by sitting or leaning on
it - it depends on the type of cushion.

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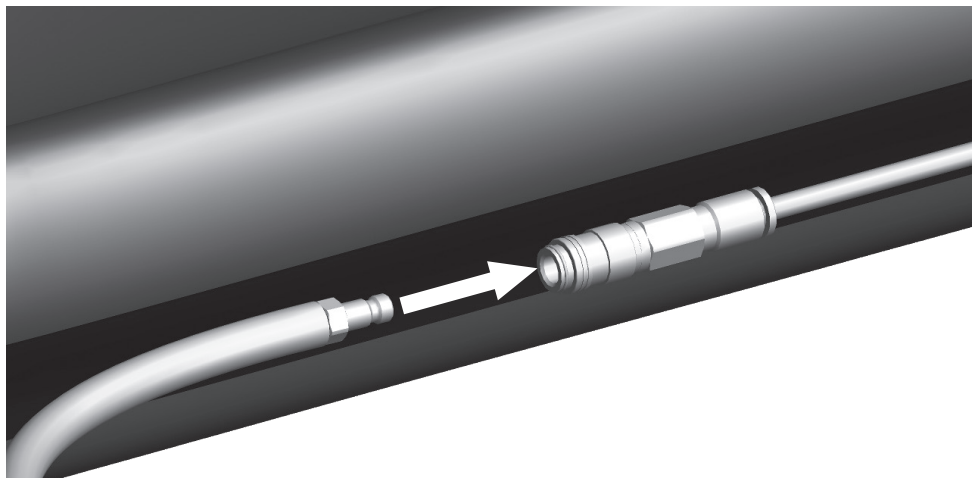
Once cushion is shaped it is necessary to
let out the air from cushion.

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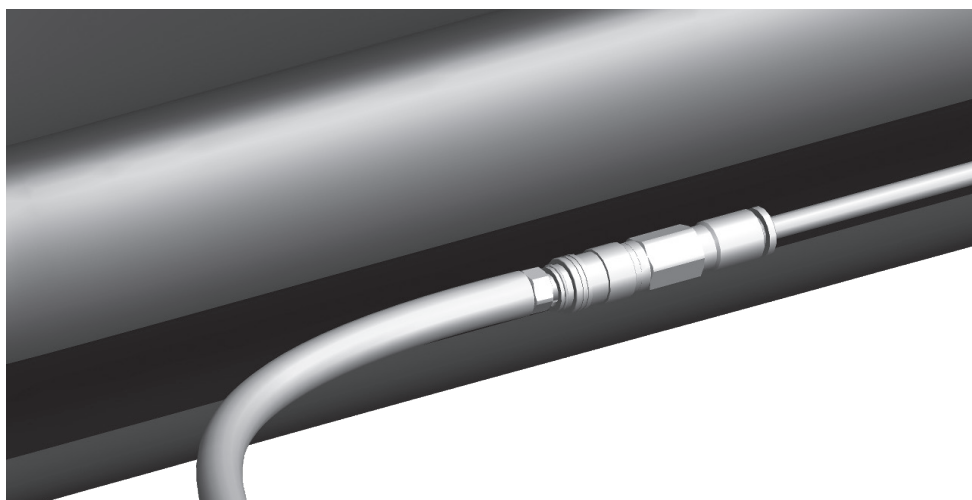If product was bought with an electric vacuum pipe it is necessary to connect these devices. Insert pipe's self-closing part into valve which is mounted on pipe.

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
Foot pump is added in case when buyer bought outsize product. Foot pump connects with the product in the same way as electric vacuum pump.

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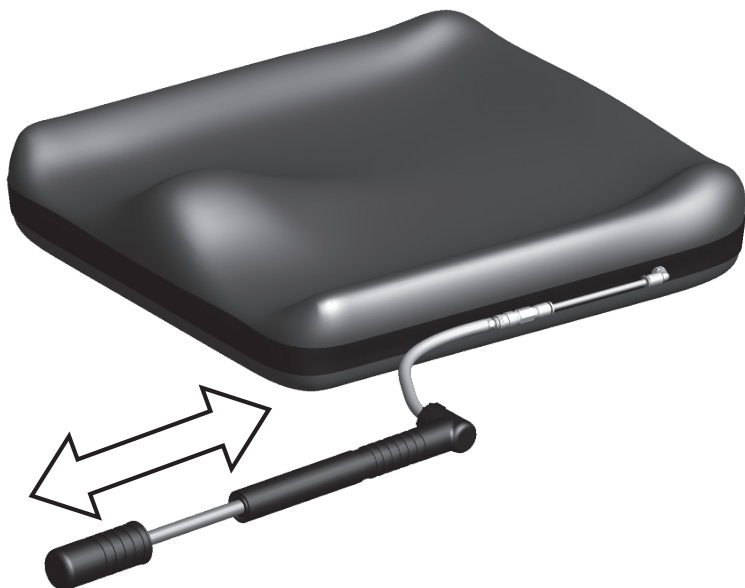As far as standard products are concerned, manufacturer adds hand pump to each product.

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
Move piston forward and backward so as to let out the air.

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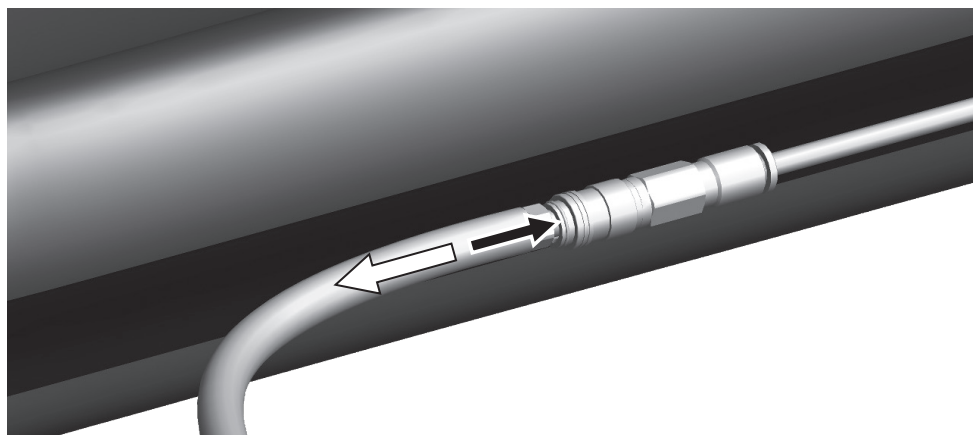
Air should be let out until it is properly hard. The shape of cushion can be easily shaped during this activity. Pump out longer so as to make cushion harder and more solid.

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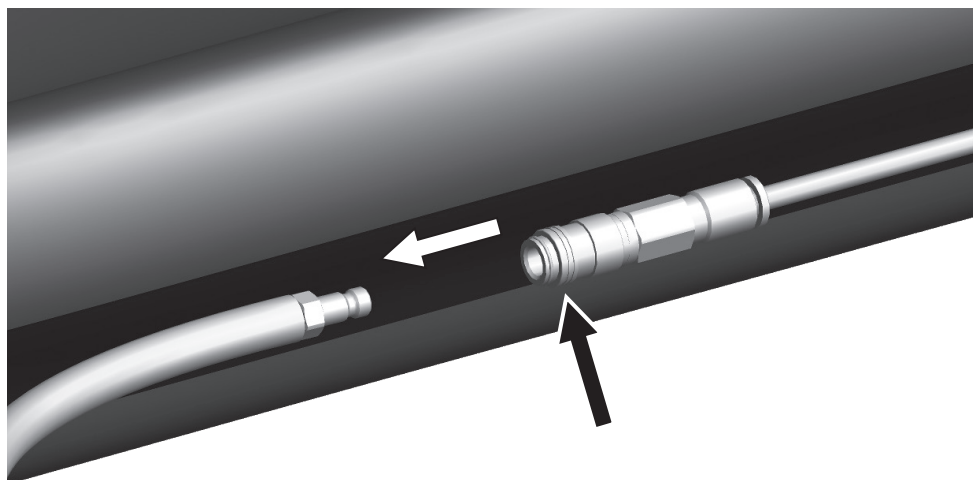Once the shape as well as hardness of cushion is done, it is necessary to disconnect vacuum pump. Valve will automatically close air supply for cushion.

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Press metal valve metal ring backwards in order to disconnect pump. Next, take out vacuum pump's pipe.

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Cushion keeps its shape for a long time. If the shape of product changed, it is important to connect vacuum pump again so as to pump out the air. It is possible to buy thermo-active cover which makes the product more comfortable to use.

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ATTENTION: To avoid falling down the granules during molding the cushions with backrest (B, C, AB, AC, etc.) please follow the steps below:

- Place the cushion on flat surface
- Spread the granules inside the cushion evenly
- vacuum the cushion a little bit so it can still be floppy and moldable
- put the cushion on its place
- start the final molding seating on it or mold it manually

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BM_001

VACUUM PUMPS

Each product is equipped with vacuum pump, depending on product size.

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CLEANING, MAINTENANCE AND REPAIR

Cushions should be regularly cleaned with the aid of dry brush or vacuum cleaner equipped with attachment made of soft bristles.

If stains are big, it is necessary to clean them with running water in a temperature up to 40 °C.

It is also possible to use detergent such as washing-up liquid or soap solution. Remember to rinse cleaned place off with running water and dry it with soft towel.

It is forbidden to use solvents: toluene or acetone.

It is forbidden to use chlorine or its derivatives.

Dry far from direct heat.

Remember to be careful when you clean banderoles which protect places that connect cushion's elements.

It is necessary to be cautious not to cut or pierce cushion; do not use any sharp tools for cleaning.

You can repair damaged cushion using "repair kit" which is included free of charge to each product. Kit consist of special neoprene super glue and few patches which you can use to seal the damage surface of the cushion.

Cushion should be sent to manufacturer in case of mechanical damages.

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MAINTENANCE VALVE

BodyMap valve is an innovative as well as easy solution used in BodyMap system. It makes it possible to control the amount of granules in BodyMap cushions. It is highly advantageous because then it is possible to adjust cushion according to your needs.

Valve is situated at the back of cushion thanks to which it is not very visible and, moreover, it is not an obstacle while using the product. What is more, it is really easy to use it – you just need to unscrew cork and either pour in or take out specific amount of granules. Additional amount of granules is packed in airtight bags that are available in two sizes. Remember that the size of bag depends on the size of BodyMap cushion.

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Please follow these steps in order to pour in/ take out granules:

1. Vacuum the cushion using the vacuum pump.

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2. Unscrew carefully valve's cork.

CAUTION! When valve is unscrewed it is forbidden to move pillow violently or press it because then granules may spill out.

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3. Take out cork which secures funnel from bag with granules. Be careful so as not to spill out its contents.

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4. Insert tightly bag's funnel into hole from cushion at a depth of 1cm. Pull bag upwards, pour granules in as much as it is needed. If you want to take out some granules from cushion, it is necessary to insert funnel into hole from cushion and then turn cushion in a way that valve is placed down. Next, take out granules as much as it is needed.

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The Velcro surface of BodyMap cushions is used for positioning of cushions on the product or fix the supporting elements on it.

First, place the BodyMap cushion on the chair, stroller etc.

In order to minimize the risk of bedsores, we recommend using a cotton cover along with each BodyMap cushion if the permanent duration of use is longer than 2 hours. For people using BodyMap cushions permanently for more than 8 hours it is recommended to use it along with thermo-elastic VISMEMO cover. Caution should be taken in subjects to people with known bedsores - prone. In such a case it is strictly recommended to you use BodyMap cushion with aeration-massaging pad for people who use BodyMap cushions non-stop longer than 8 hours !

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