REH4MAT
36-060 Głogów Małopolski, ul. Piaski 47, POLAND
Adres korespondencyjny/Mailing address:
ul. Truskawkowa 17, Widna Góra, 37-500 Jarosław, POLAND
tel. +48 (016) 621 41 35, fax.: +48 (016) 621 42 13
E-mail address/Adres mailowy:
biuro@reh4mat.com
www.reh4mat.com; www.probraces.com
The company holds the ISO 13485 International
quality management system certificate
INSTRUCTION: 14.06.2018r.

OPIS PRODUKTU:

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PRODUCT DESCRIPTION:

The waterproof bed pad is intended to protect mattress from getting wet. The product could be used instead of the normal bed cover and as an additional protection under the bed cover. The product is used in lying people suffering from urinary incontinence. In standard conditions, the product may be also used to prevent a patient from diseases that are caused by infections and impurities. Thanks to its waterproof feature, the cover is a protective barrier that prevents a patient's skin from pathogenic micro-organisms, impurities and liquids.

The waterproof sheet is intended to protect mattress from getting wet. The product could be used instead of the normal bed cover and as a additional protection under the bed cover. The product is used in lying people suffering from urinary incontinence.

Our products are appreciated by specialists (physicians, physical therapist) and patients. REH4MAT products are innovative and are carefully manufactured of the high quality materials. Our objective is customer's satisfaction. In case of any questions feel free to contact us by e-mail: biuro@reh4mat.com.

IMPORTANT:

- Proper use of the product requires diagnosis by physician or physical therapist.
- It is necessary to control using the product, especially in children, people with memory dysfunction, psychiatric and people with sensory disturbances.
- The skin in place, where the product touches should be clean.
- Is it necessary to control the skin's condition (especially in heels and feet) and in case of skin lesions occurring is it necessary to contact with physician.
- In case of skin lesions (epidermis scrapes, scatches or eczema) using of the product is allowed after dressing applying.

CARE AND MAINTENANCE:

- Wash in temp. 60°C,
- Slicce carefully,
- Do not iron,
- Do not chlorinate,
- Do not dry clean.
- Protect from moisture,
- Protect from high temperatures,
- Dry away from sources of warmth,
- Do not swirl.

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REH4MAT
MANUFACTURER OF MODERN ORTHOPAEDIC BRACES

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BEFORE USING A PAD AND SHEET, IT IS NECESSARY TO FAMILIARIZE YOURSELF WITH THE CONTENT OF THIS INSTRUCTION AND CHECK IF THE ELEMENTS ARE NOT WORN OUT. SAVE THE PRODUCT INFORMATION, TO BE ABLE TO REREAD THE INSTRUCTION LATER.

PAIN IS ALWAYS THE SIGN OF WARNING. IF THE PAIN MAINTAINS OR INCREASES IT IS ADVISABLE TO CONSULT YOUR DOCTOR OR PHYSICAL THERAPIST.

FOR SINGLE PATIENT USE ONLY. USING OF THE DEVICE NOT ACCORDING TO THE MANUAL MAY CAUSE SERIOUS COMPLICATIONS.

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