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The waterproof sheet is intended to protect mattress from getting wet. The product could be used instead of the normal bed cover and as a additional protection under the bed cover. The product is used in lying people suffering from urinary incontinence.

Our products are appreciated by specialists (physicians, physical therapist) and patients. REH4MAT products are innovative and are carefully manufactured of the high quality materials. Our key objective is customer's satisfaction. In case of any questions feel free to contact us by e-mail: biuro@reh4mat.com.

IMPORTANT:

- Proper use of the product requires diagnosis by physician or physical therapist.
- It is necessary to control using the product, especially in children, people with memory dysfunction, psychiatric and people with sensory disturbances.
- The skin in place, where the product touches should be clean.
- Is it necessary to control the skin's condition (especially in heels and feet) and in case of skin lesions occurring is it necessary to contact with physician.
- In case of skin lesions (epidermis scrapes, scatches or eczema) using of the product is allowed after dressing applying.

CARE AND MAINTENANCE:

- Wash in temp. 60°C,
- Sluice carefully,
- Do not iron,
- Do not chlorinate,
- Do not dry clean.
- Protect from moisture,
- Protect from high temperatures,
- Dry away from sources of warmth,
- Do not swirl.

WATERPROOF SHEET:

- Relieves the effects of urinary incontinence.
- They are therapeutic support of incontinence's treatment for medication or positional therapy.
- They are useful in case of neurological disorders (Parkinson's disease, SM), urological conditions (bladder cancer, kidney stone) or while climacteric.
- Waterproof sheet are made of breathable (vapour-permeable) fabric and can be used by people with bedsores risk.
- They can be used together with hospital mattresses as reduce risk of urine ad feces absorption, oxygante the skin and prevent against the bedsores.
- Thanks to its waterproof feature, the cover is a protective barrier that prevents a patient's skin from pathogenic micro-organisms, impurities and liquids.

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BEFORE USING A PAD AND SHEET, IT IS NECESSARY TO FAMILIARIZE YOURSELF WITH THE CONTENT OF THIS INSTRUCTION AND CHECK IF THE ELEMENTS ARE NOT WORN OUT. SAVE THE PRODUCT INFORMATION, TO BE ABLE TO REREAD THE INSTRUCTION LATER.

PAIN IS ALWAYS THE SIGN OF WARNING. IF THE PAIN MAINTAINS OR INCREASES IT IS ADVISABLE TO CONSULT YOUR DOCTOR OR PHYSICAL THERAPIST.

FOR SINGLE PATIENT USE ONLY. USING OF THE DEVICE NOT ACCORDING TO THE MANUAL MAY CAUSE SERIOUS COMPLICATIONS.

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