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

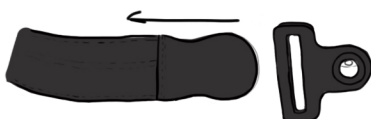
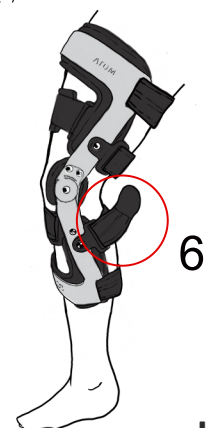
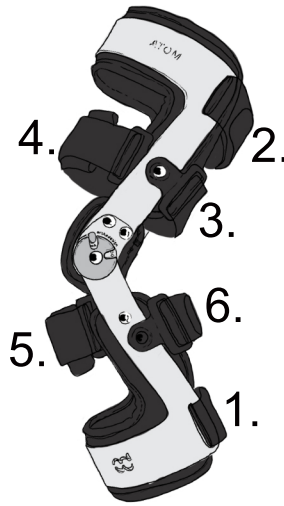
1. Unblock the side splints to reach the maximum range of flexion and extension of the brace.

2. Unbuckle the particular circumferential straps.

3. To avoid tangling the straps, fasten the velcro to the strap again when it is removed from the buckle.


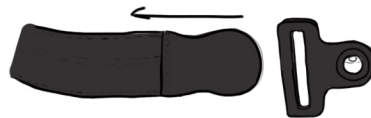
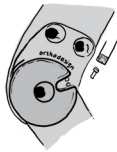
Unscrew the angular holes

4. Keep the lower limb in the 30-degree flexion and put the brace on your knee.

5. Twine the strap through the polyamid buckle and fasten the calf strap (no. 1).

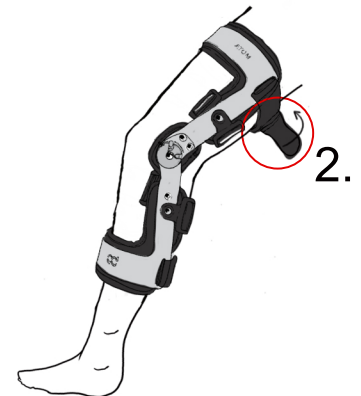
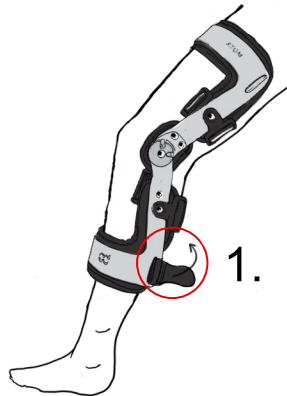
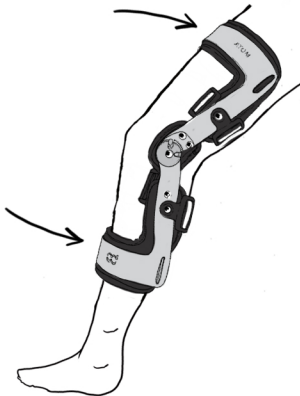
6. Center the hinges exactly along the knee axis. Twine the strap through the polyamid buckle and fasten the back thigh strap (no. 2).

7. Twine the strap through the polyamid buckle and fasten the back thigh strap (no. 3) and the front thigh strap (no. 4). Attention: the inappropriate tension of a back thigh strap may result in moving the hinges to the back. That is why, the tension of a strap should be adequate to the final hinge position - the hinges must be placed exactly along the knee axis.


8. Flex the knee joint to approx. 100-degree and fasten the front strap (no. 5) tightly. The soft shin pad should be placed in the middle of the shin.

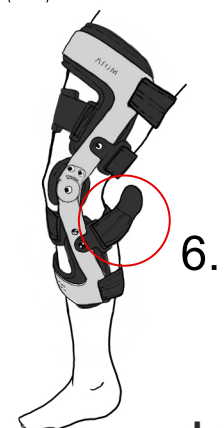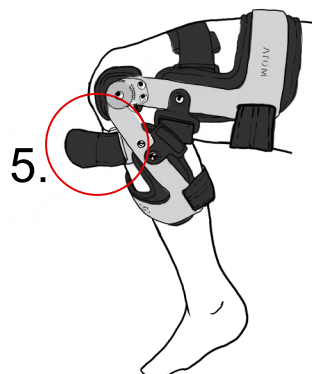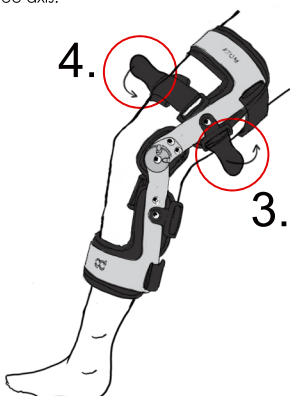9. Straighten and extend your lower limb. Fasten back shin strap (no.6).

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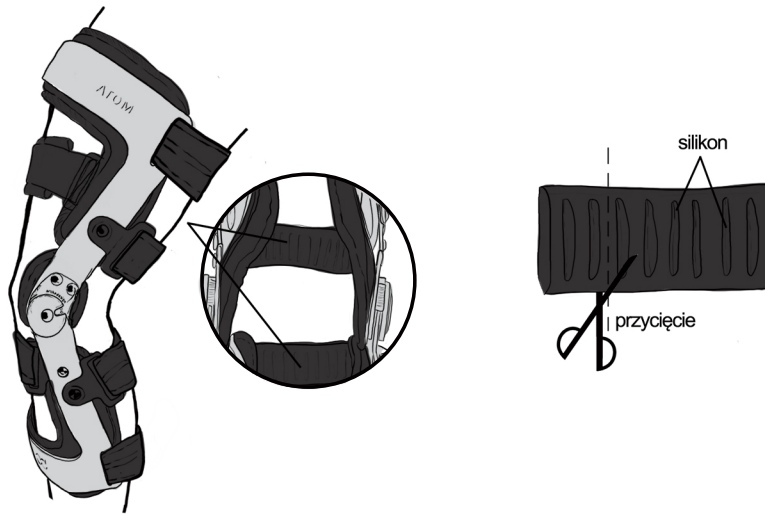
ATTENTION!

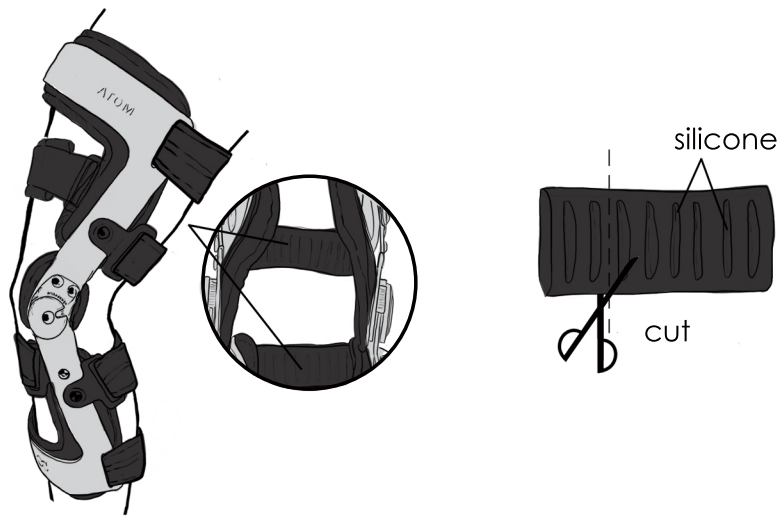1. The brace is equipped with the system of anti-sliding soft covers with the self-adhesive function. They always need to be set so that the middle part of them was placed exactly along the knee axis. To make the brace perfectly fitted, it is possible to shorten the covers. The shortening of the straps do not make the brace damaged and there is no need to protect it additionally.

2. The circumferential straps of ATOM 1R brace are equipped with with special Velcro fastening. Moreover, on the inside side of the straps there is regulative buckle that lets a patient set the ideal length of circumferential strap according to the individual needs.

3. The internal soft lining of the brace prevent from brace sliding. If there is a need, you can detach it from the brace frame and hand-wash it carefully using the warm water. Then wash the lining out and shrivel. Do not bleach, do not squeeze, do not spin, do not wash in washing machine, do not dry near radiators, etc. When the lining is dry, put it carefully to the brace frame again.