POLSKI

ENGLISH

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KNEE BRACE UNDERSLEEVE

ATTENTION: BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTION COMPLETELY AND CAREFULLY AND CHECK IF PRODUCT'S ELEMENTS AND SEAMS ARE SEEDY. ONLY UNDAMAGED PRODUCT AND ITS CORRECT APPLICATION ARE VITAL TO THE PROPER FUNCTIONING OF IT. SAVE THE PRODUCT INFORMATION, TO BE ABLE TO REREAD THE INSTRUCTION LATER.

PRODUCT DESCRIPTION:

The knee brace undersleeve is intended for our functional knee braces. Using of our knee brace undersleeve increases of your comfort, prevents against the skin abrasions and provides stabilization in knee joint.
Reh4Mat products are innovative and are carefully manufactured of the high quality materials. Our key objective is customer's satisfaction.
In case of any questions feel free to contact with us by e-mail: biuro@reh4mat.com.

IMPORTANT:

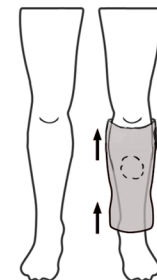
- Proper use of the product requires diagnosis by physician or physical therapist.
- Optimal product functioning can only be achieved by selecting the right size. The Reh4Mat product must be fitted by an authorized vender, physician or educated physiotherapist.
- It is necessary to control using the product, especially in children, people with memory dysfunction, psychiatric and people with sensory disturbances.
- The skin in place, where the product touches should be clean (the creams or ointments should not be used).
- Is it necessary to control the skin's condition and in case of skin lesions occurring is it necessary to contact with physician.
- In case of skin lesions (epidermis scrapes, scathes or eczema) using of the product is allowed after dressing applying.
- Chronic using the product may cause limitation of the active and passive movement in joints. If there are no contraindications, it is recommended to exercise several times during the day. Exercises are selected individually for each patient by a master's degree or technician rehabilitation as physician has recommended.
- In case of varicose veins, Deep Vein Thrombosis (DVT), lower limb oedema the using

of product is allowed only after contact with physician.

- Use of the product during the sleeping should be done only if this was prescribed by your doctor.

CARE AND MAINTENANCE:

- Wash in temperature of 30 C,
- Sluice carefully,
- Do not iron,
- Do not chlorinate,
- Do not dry clean,
- Do not tumble dry,
- Protect from moisture,
- Protect from high temperatures,
- Dry away from sources of warmth.

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ATTENTION: FOR SINGLE PATIENT USE ONLY. USING OF THE PRODUCT NOT ACCORDING TO THE MANUAL MAY CAUSE SERIOUS COMPLICATIONS.