POLSKI

ENGLISH

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ANTI-BEDSORE MATTRESS

ATTENTION: BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTION COMPLETELY AND CAREFULLY AND CHECK IF MATTRESS'S ELEMENTS AND SEAMS ARE SEEDY. ONLY UNDAMAGED DEVICE AND ITS CORRECT USING ARE VITAL TO THE PROPER FUNCTIONING OF IT. SAVE THE PRODUCT INFORMATION, TO BE ABLE TO REREAD THE INSTRUCTION LATER.

PRODUCT DESCRIPTION:

Mattress is specially designed to provide user comfort. Mattress is an excellent solution for anti bedsores therapy. Reh4Mat products are innovative and are carefully made of the high quality materials. Our key objective is customer's satisfaction. In case of any questions feel free to contact with us by e-mail: biuro@reh4mat.com.

IMPORTANT:

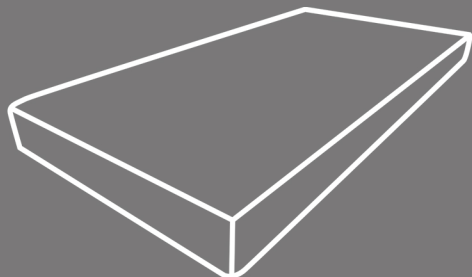
- In case of skin lesions occurring and allergic reaction it is necessary to contact with physician.

CARE AND MAINTENANCE:

- Do not wash,
 - Do not iron,
 - Do not chlorinate,
 - Do not dry clean,
 - Do not tumble dry,
 - Protect from moisture,
 - Protect from high temperatures,
 - Dry away from sources of warmth
- To disinfect the cover spray the surface of the device with disinfectant, leave it for disinfectant's activation, then wash it by water and dry it. The cover should be disinfected by disinfectant. The active substances and their maximum concentration: 2-propanol (35%), 1-propanol (25%), ethanol (10%) alkyl dimethyl benzyl ammonium chloride (0,2%), glucoprotamine(25%).

ATTENTION: Mattress should be used with a cover.

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ATTENTION: USING OF THE DEVICE NOT ACCORDING TO THE MANUAL MAY CAUSE SERIOUS COMPLICATIONS.