
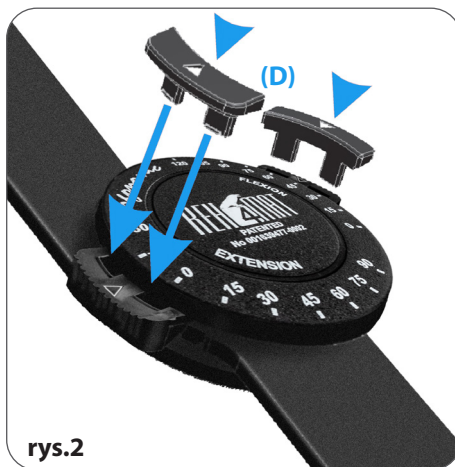


INSTRUKCJA USTAWIENIA KĄTA ZGIĘCIA I WYPROSTU W SZYNIE 1R ERGONOMIC

Orteza została wyposażona w szynę boczną, która posiada blokadę kąta **zgięcia (A)** w zakresie: 0°, 15°, 30°, 45°, 60°, 75°, 90°, 105°, 120° i ograniczeniem kąta **wyprostu (B)** w zakresie: -30°, -15°, 0°, 15°, 30°, 45°, 60°, 75°, 90°.

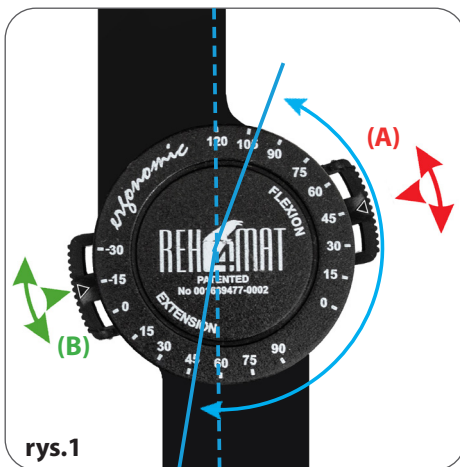

1. Ustaw ortezę w przybliżonym położeniu, w którym chcesz ograniczyć **zgięcie (A)** lub **wyprost (B)** (rys.1)
2. Wciskając ograniczniki **(A)** i **(B)** przemieść je po obwodzie zegara w pożądane miejsce z opisanym kątem (rys.1)
3. Zablokuj ograniczniki specjalnymi blokadami **(D)** dostarczonymi z szyną (rys.2)
4. Sprawdź poprawność działania i zakres ruchów zegara

ZAKRES RUCHÓW I JEGO WARTOŚCI KĄTOWE USTALA LEKARZ PROWADZĄCY!

UWAGA: Jeżeli orteza wyposażona jest w dwie szyny boczne, regulacji należy dokonać osobno na obydwu szynach, ustawiając ten sam kąt zgięcia lub wyprostu. Szyny boczne nie wymagają konserwacji



REH4MAT sp. z o.o.

ul. Zenitowa 5a, 35-301 Rzeszów | Polska

tel. +48 16 621 42 20



tel. +48 16 621 41 35

e-mail: biuro@reh4mat.com

www.reh4mat.com

HOW TO SET UP THE FLEXION AND EXTENSION IN 1R ERGONOMIC SPLINT - INSTRUCTION

This brace is equipped with two splitus with ROM adjustment **flexion (A)** in the scope of: 0°, 15°, 30°, 45°, 60°, 75°, 90°, 105°, 120° and **extension (B)** in the scope of: -30°, -15°, 0°, 15°, 30°, 45°, 60°, 75°, 90°.

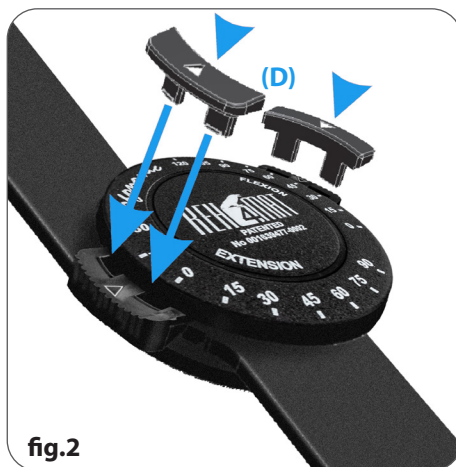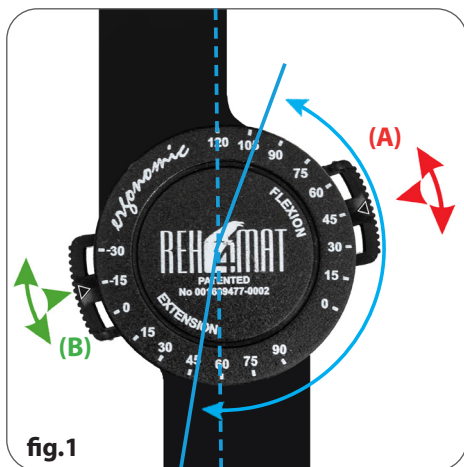

1. Place the brace in the position in which you want to adjust the **flexion (A)** or **extension (B)** (fig.1)
2. By pressing the buttons (A) and (B) - delimiters to the center of the splint move them around to the desired place with the prescribed angle (fig.1)
3. Look buttons with the special lockers (D) included in this kit (fig.2)
4. Check the correctness of operation and the range of clock movements

THE ANGLE OF FLEXION AND EXTENSION IS SPECIFIED BY THE DOCTOR OR PHYSIOTHERAPIST!

ATTENTION: If the brace is equipped with 2 side splints, adjustment is done on each side splint separately setting the same angle of flexion and extension. Side splints do not require maintenance



REH4MAT sp. z o.o.

ul. Zenitowa 5a, 35-301 Rzeszów | Polska

tel. +48 16 621 42 20

tel. +48 16 621 41 35

e-mail: biuro@reh4mat.com

www.reh4mat.com