


INSTRUKCJA USTAWIENIA ZAKRESU RUCHOMOŚCI KĄTA ZGIĘCIA I WYPROSTU W SZYNYE 1RP

Orteza została wyposażona w szynę boczną, która posiada blokadę kąta **zgięcia (A)** w zakresie: 30°, 50°, 70° oraz blokadę **wyprostu (B)** w zakresie: 0°, 20°.

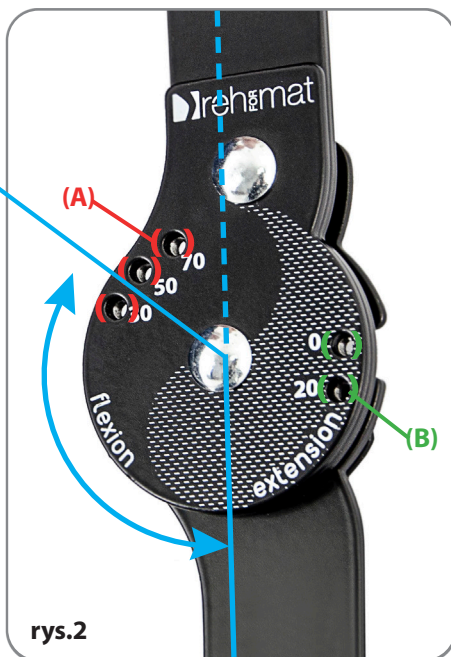

1. Przed ustawieniem szyny, usuń wszystkie wkręty blokujące umieszczone w zegarze (**rys.1**)
2. Ustaw ortezę w przybliżonym położeniu, w którym chcesz ograniczyć **zgięcie/wyprost** (**rys.2**)
UWAGA! Ze względów bezpieczeństwa konieczne jest zablokowanie szyny w pozycji wyprostu (B). Brak blokady kąta wyprostu może skutkować niekontrolowanym przeprostem stawu kolanowego.
3. W otwór oznaczony odpowiednią wartością kąta wkręć od spodu dołączone śruby blokujące oraz sprawdź poprawność działania i zakres ruchów zegara (**rys.2**)

ZAKRES RUCHÓW I JEGO WARTOŚCI KĄTOWE USTALA LEKARZ PROWADZĄCY!

UWAGA: Jeżeli orteza wyposażona jest w dwie szyny boczne, regulacji należy dokonać osobno na obydwu szynach, ustawiając ten sam kąt zgięcia lub wyprostu. Szyny boczne nie wymagają konserwacji



REH4MAT sp. z o.o.

ul. Zenitowa 5a, 35-301 Rzeszów | Polska

tel. +48 16 621 42 20


tel. +48 16 621 41 35

e-mail: biuro@reh4mat.com

www.reh4mat.com

HOW TO SET UP THE FLEXION AND EXTENSION IN 1RP SPLINT - INSTRUCTION

This brace is equipped with the following ROM adjustment **flexion (A)** in the scope of: 30°, 50°, 70° and **extension (B)** in the scope of: 0°, 20°.

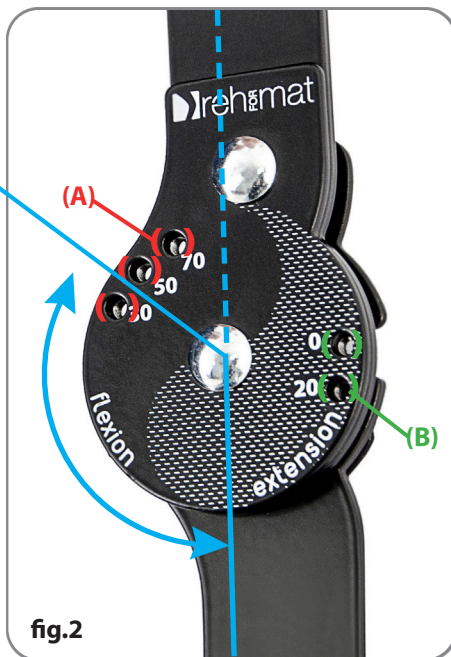

1. Before setting up the splint, remove the screws from it (fig.1)
2. Place the brace in the position in which you want to adjust the **flexion/extension** (fig.2)
ATTENTION! For safety reasons, it is necessary to lock the splint in its extension position (B).
Failure to block the extension angle may result in uncontrolled hyperextension of the knee joint.
3. Place the screw in the hole setting appropriate angle and check the if it works correctly (fig.2)

THE ANGLE OF FLEXION AND EXTENSION IS SPECIFIED BY THE DOCTOR OR PHYSIOTHERAPIST!

ATTENTION: If the brace is equipped with 2 side splints, adjustment is done on each side splint separately setting the same angle of flexion and extension. Side splints do not require maintenance



REH4MAT sp. z o.o.

ul. Zenitowa 5a, 35-301 Rzeszów | Polska

tel. +48 16 621 42 20

tel. +48 16 621 41 35

e-mail: biuro@reh4mat.com

www.reh4mat.com