


## INSTRUKCJA USTAWIENIA ZAKRESU RUCHOMOŚCI KĄTA ZGIĘCIA I WYPROSTU W SZYNE 2RA PRECISION

Orteza została wyposażona w szynę boczną, która posiada blokadę kąta **zgięcia (A)** w zakresie: 0°, 30°, 55°, 75°, 95°, 110° oraz blokadę **wyprostu (B)** w zakresie: 15°, 75°.

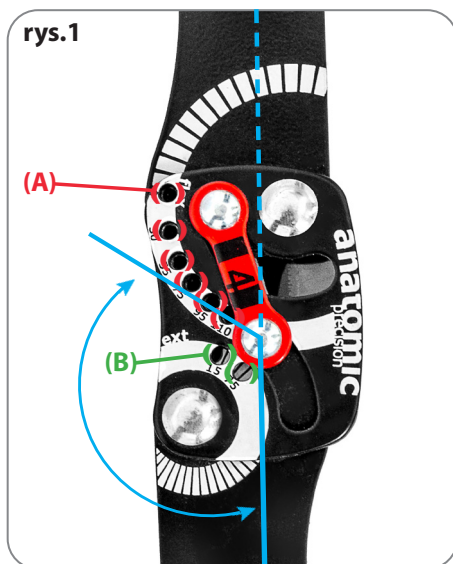

1. Ustaw ortezę w przybliżonym położeniu, w którym chcesz ograniczyć **zgięcie/wyprost (rys.1)**
2. Wkręć wkręt/wkręty w otwór oznaczony odpowiednią wartością kąta i sprawdź poprawność działania oraz zakres ruchów zegara (**rys.2**)

**ZAKRES RUCHÓW I JEGO WARTOŚCI KĄTOWE USTALA LEKARZ PROWADZĄCY!**

**UWAGA:** Jeżeli orteza wyposażona jest w dwie szyny boczne, regulacji należy dokonać osobno na obydwu szynach, ustawiając ten sam kąt zgięcia lub wyprostu. Szyny boczne nie wymagają konserwacji



REH4MAT sp. z o.o.

ul. Zenitowa 5a, 35-301 Rzeszów | Polska

tel. +48 16 621 42 20


tel. +48 16 621 41 35

e-mail: [biuro@reh4mat.com](mailto:biuro@reh4mat.com)

[www.reh4mat.com](http://www.reh4mat.com)

## HOW TO SET UP THE FLEXION AND EXTENSION IN 2RA PRECISION SPLINT - INSTRUCTION

The brace is equipped with a side splint with a bending **flexion (A)** in the scope of: 0°, 30°, 55°, 75°, 95°, 110° and **the extension (B)** in the scope of: 15°, 75°.

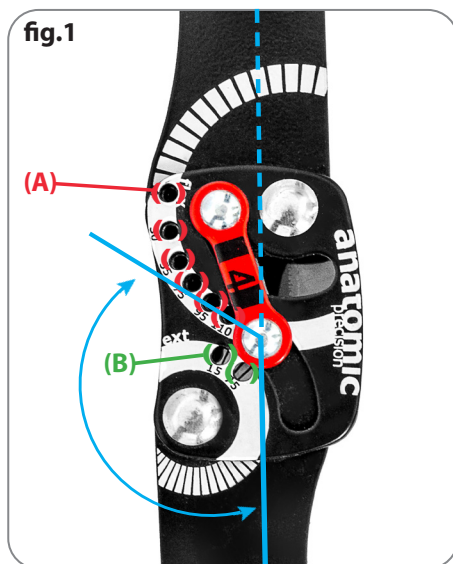

1. Position the brace in the approximate position where you want to limit the **flexion/extension** (fig.1)
2. Drive the screw/bolts into the hole marked with the appropriate angle value and check the correct operation and the range of clock movements (fig.2)

**THE ANGLE OF FLEXION AND EXTENSION IS SPECIFIED BY THE DOCTOR OR PHYSIOTHERAPIST!**

**ATTENTION:** If the brace is equipped with 2 side splints, adjustment is done on each side splint separately setting the same angle of flexion and extension. Side splints do not require maintenance



REH4MAT sp. z o.o.

ul. Zenitowa 5a, 35-301 Rzeszów | Polska

tel. +48 16 621 42 20

tel. +48 16 621 41 35

e-mail: [biuro@reh4mat.com](mailto:biuro@reh4mat.com)

[www.reh4mat.com](http://www.reh4mat.com)