REH4MAT Sławomir Wroński
36-060 Głogów Małopolski, ul. Paski 47, POLAND
Adres korespondencji/Mailing address:
ul. Truskawkowa 17, Widna Góra, 37-500 Jarosław, POLAND
tel. +48 (016) 621 42 20
E-mail address/Adres mailowy:
biuro@reh4mat.com
www.reh4mat.com; www.probraces.com
The company holds the ISO 13485 International
quality management system certificate
INSTRUCTION: 12.07.2019

OPIS PRODUKTU:

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PRODUCT DESCRIPTION:

The brace is an excellent solution for effective therapy in case of injury or other conditions. Braces are appreciated by specialists (physicians, physical therapist) and patients.
Reh4Mat products are innovative and are carefully manufactured of the high quality materials. Our key objective is customer's satisfaction.
In case of any questions feel free to contact with us by e-mail: biuro@reh4mat.com.

IMPORTANT:

- Proper use of the brace requires diagnosis by physician or physical therapist.
- Optimal brace functioning can only be achieved by selecting the right size. The Reh4Mat brace must be fitted by an authorized vender, physician or educated psychotherapist.
- It is necessary to control using the brace, especially in children, people with memory dysfunction, psychiatric and people with sensory disturbances.
- The skin in place, where the brace touches should be clean (the creams or ointments should not be used).
- Is it necessary to control the skin's condition (especially in heels and feet) and in case of skin lesions occurring is it necessary to contact with physician.
- In case of skin lesions (epidermis scrapes, scathes or eczema) using of the braces is allowed after dressing applying.
- Chronic using the brace may cause limitation of the active and passive movement in joints. If there are no contraindications, it is recommended to exercise several times during the day. Exercises are selected individually for each patient by a master's degree or technician rehabilitation as physician has recommended.
- In case of varicose veins, Deep Vein Thrombosis (DVT), lower limb or upper limb oedema the using of brace is allowed only after contact with physician.
- Use of the brace during the night should be done only if this was prescribed by your doctor.

CARE AND MAINTENANCE:

- Wash by hands in warm water with soap in temperature of 30 C.
- Sluice carefully.
- Do not iron.
- Do not chlorinate.
- Do not dry clean.
- Do not tumble dry.
- Protect from moisture.
- Protect from high temperatures.
- Dry away from sources of warmth.

INDICATION:

Torso support provides stabilization and relieving the trunk and spine. The support provides compression and stabilization of thoracic, lumbar and spinal spine. It is used in case of rheumatic, neurological and orthopedic disorders and injuries.

IF THE DEVICE HOLDS LATERAL SPLINTS, SHELLS AND IT IS POSSIBLE TO DISMANTLE THEM WITHOUT ANY TOOLS, BEFORE WASHING REMOVE THE SPLINTS.

IF A BRACE IS EQUIPPED WITH SIDE SPLINTS WITH ROM ADJUSTMENT, THE ATTACHED INSTRUCTION OF USING THE RANGE OF MOTION IN THE SPLIT MUST BE ABSOLUTELY RESPECTED.

BEFORE USING A BRACE, PLEASE FAMILIARIZE YOURSELF WITH THE WEARING INSTRUCTION ATTACHED AS PICTOGRAPHS TO EACH DEVICE.

BEFORE USING A BRACE, IT IS NECESSARY TO FAMILIARIZE YOURSELF WITH THE CONTENT OF THIS INSTRUCTION AND CHECK IF THE ELEMENTS ARE NOT WORN OUT. SAVE THE PRODUCT INFORMATION, TO BE ABLE TO REREAD THE INSTRUCTION LATER.

PAIN IS ALWAYS THE SIGN OF WARNING. IT THE PAIN MAINTAINS OR INCREASES IT IS ADVISABLE TO CONSULT YOUR DOCTOR OR PHYSICAL THERAPIST.

FOR SINGLE PATIENT USE ONLY. USING OF THE DEVICE NOT ACCORDING TO THE MANUAL MAY CAUSE SERIOUS COMPLICATIONS.

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