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INDICATION:

Underarm crutch is an excellent solution for people who need support in walking. Product should be used by people suffering from lower limbs weakness or paralysis, after amputation or other neurological or orthopaedic conditions (fractures, injuries). It is highly recommended for elderly people with weak legs. Our products are appreciated by specialists (physicians, physical therapist) and patients. REH4MAT products are innovative and are carefully manufactured of the high quality materials. Our key objective is customer's satisfaction.

In case of any questions feel free to contact us by e-mail: biuro@reh4mat.com.

IMPORTANT:

- Proper use of the product requires diagnosis by physician or physical therapist.
- It is necessary to control using the product, especially in children, people with memory dysfunction, psychiatric and people with sensory disturbances.
- The skin in place, where the crutch touches should be clean.
- Is it necessary to control the skin's condition (especially in heels and feet) and in case of skin lesions occurring is it necessary to contact with physician.
- Chronic using the cushions may cause limitation of the active and passive movement in joints. If there are no contraindications, it is recommended to exercise several times during the day. Exercises are selected individually for each patient by a master's degree or physical therapy technician as physician has recommended.

ADJUSTMENT:

1. Handle setting:
 - unscrew the wing nut,
 - slide out the screw from lateral laths and handle,
 - set the handle to the measured distance from armpit to hand with gentle elbow flexion,
 - slip back the screw to the holes made in lateral laths and handle,
 - put on the wing nut.
2. Height setting:
 - unscrew two wing nuts,
 - slide out two screws from lateral laths and middle element,
 - adjust the height to the user,
 - slip back the two screws and embed them by wing nuts.

CARE AND MAINTENANCE:

- Armpit should be cleaned with a moist cloth,
- Do not wash,
- Do not iron,
- Do not chlorinate,
- Do not swirl,
- Do not dry clean,
- Do not tumble dry,
- Protect from moisture,
- Protect from high temperatures,
- Dry away from sources of warmth.

ATTENTION:

PLEASE BE CAREFUL WHILE USING THE CRUTCH ON AN UNSAFE SURFACE: WET, ICY, SANDY.
THE PRODUCT IS INTENDED FOR PEOPLE WITH A MAXIMUM WEIGHT OF 100 KG.
ADJUSTMENT RANGE FROM FLOOR TO THE UNDERARM IS 105 CM - 150 CM.

BEFORE USING A CUSHION, IT IS NECESSARY TO FAMILIARIZE YOURSELF WITH THE CONTENT OF THIS INSTRUCTION AND CHECK IF THE ELEMENTS ARE NOT WORN OUT. SAVE THE PRODUCT INFORMATION, TO BE ABLE TO REREAD THE INSTRUCTION LATER.
PAIN IS ALWAYS THE SIGN OF WARNING. IF THE PAIN MAINTAINS OR INCREASES IT IS ADVISABLE TO CONSULT YOUR DOCTOR OR PHYSICAL THERAPIST.

FOR SINGLE PATIENT USE ONLY. USING OF THE DEVICE NOT ACCORDING TO THE MANUAL MAY CAUSE SERIOUS COMPLICATIONS.

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