

KĄT ZGIĘCIA LUB WYPROSTU USTALA LEKARZ PROWADZĄCY LUB REHABILITANT

Niniejsza orteza została wyposażona w szyny boczne z możliwością ograniczenia kąta zgięcia w 0°, 15°, 30°, 45°, 60°, 75°, 90°, 105°, 120°, ograniczeniem kąta wyprost w 90°, 75°, 60°, 45°, 30°, 15°, 0°, oraz przeprostu w -15°, -30°.

1. Ustawić ortezę w przybliżonym położeniu w którym chcemy ograniczyć zgięcie lub wyprost.
2. Przyciskając ograniczniki do środka przegubu przemieścić je po obwodzie w pożądane miejsce z opisanym kątem.
3. Zablokować ograniczniki specjalnymi blokadkami dostarczonymi z szyną.

UWAGA:
PRZED USTAWIENIEM BLOKADY KĄTA ZGIĘCIA LUB WYPROSTU, ZALECA SIĘ ZAŁOŻYĆ WYRÓB NA KOŃCZYNE.
Szyny boczne nie wymagają konserwacji.


Termin ważności: NIE OKREŚLA SIĘ 📅 2013
Okres gwarancji: 24 MIESIĄCE OD DATY ZAKUPU

Konsultant Med: tel. 508 382 509



Podstawa dopuszczenia do obrotu:
Produkt został zarejestrowany w Urzędzie Rejestracji Produktów Leczniczych, Wyrobów Medycznych i Produktów Biobójczych w Wyrob klasy I

REH4MAT Sławomir Wroński
ul. Piaski 47, 36-060 Głogów Młp., Polska
tel. +48 16 621 42 20; fax +48 16 621 42 13
e-mail: biuro@reh4mat.com, www.reh4mat.com

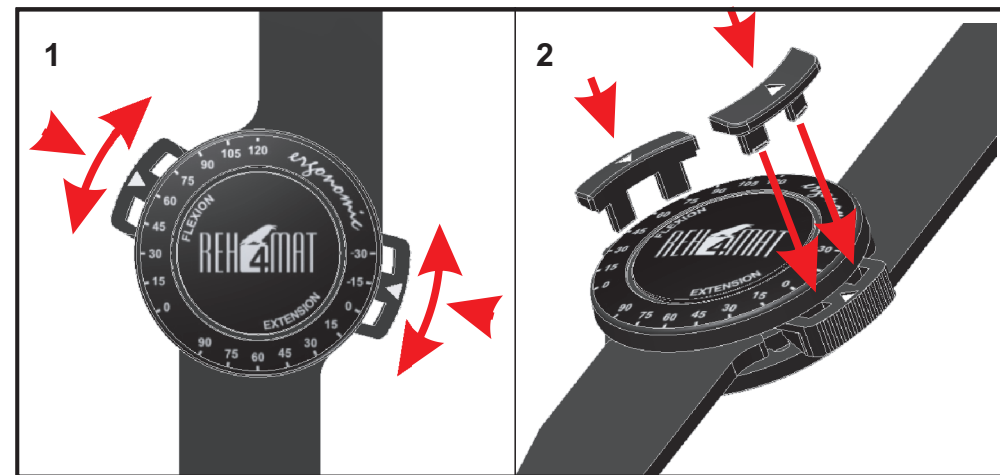

THE ANGLE OF BENDING AND STRAIGHTENING IS SPECIFIED BY THE DOCTOR OR PHYSIOTHERAPIST

FLEXION: 0°, 15°, 30°, 45°, 60°, 75°, 90°, 105°, 120°. EXTENTION: 90°, 75°, 60°, 45°, 30°, 15°, 0°, -15°, -30°.

1. Place the support in the position in which you want to stabilize the flection (extension).
2. By pressing the buttons - delimiters (1) to the center of the splint move them around to the desired place with the prescribed angle.
3. Lock buttons with the special lockers (2) that came with splint.

ATTENTION:

First put the device on the limb than adjust the angle of bending and straightening.

If the orthosis is equipped with 2 side splints, adjustment is done on each side splint separately, setting the same angle of bending and straightening on both splints.

Side splints do not require maintenance.

Expiry date - IS NOT DETERMINED. 📅 2013

Guarantee period - 24 months from the date of purchasing.



Admitted on the market: product registered in the Department of Registration of Healing Products, Medical Products and Biocide Products.
Medical product first class

REH4MAT Sławomir Wroński
ul. Piaski 47, 36-060 Głogów Młp., Poland
tel. +48 16 621 42 20; fax +48 16 621 42 13
e-mail: biuro@reh4mat.com, www.reh4mat.com