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PRODUCT DESCRIPTION:

Rehabilitation wedge is designed for treatment and physical therapy. It is used in sitting or lying positions for supporting the legs and reducing the swelling and muscles contractions and spasms. The wedge is used also in positional therapy and reduces the risk of bedsores.

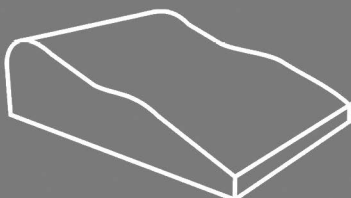
IMPORTANT:

- Proper use of the product requires diagnosis by physician or physical therapist.
- Optimal wedge and ring functioning can only be achieved by selecting the right size. The REH4MAT products must be fitted by an authorized vender, physician or educated physiotherapist.
- It is necessary to control using the product, especially in children, people with memory dysfunction, psychiatric and people with sensory disturbances.
- The skin in place, where the wedge or ring touches should be clean.
- Is it necessary to control the skin's condition (especially in heels and feet) and in case of skin lesions occurring is it necessary to contact with physician.
- In case of skin lesions (epidermis scrapes, scatches or eczema) using of the product is allowed after dressing applying.
- Chronic using the products may cause limitation of the active and passive movement in joints. If there are no contraindications, it is recommended to exercise several times during the day. Exercises are selected individually for each patient by a master's degree or physical therapy technician as physician has recommended.
- In case of varicose veins, Deep Vein Thrombosis (DVT), lower limb or upper limb oedema the using of wedges and rings is allowed only after contact with physician.

CARE AND MAINTENANCE:

- The cotton cover can be washed in washing machine in temperature of 40°C.
 - The Sanmed C463 cover should be cleaned regularly with mild detergent with a sponge or soft brush, after that it should be wiped by damp cloth and then wiped dry. To disinfection the cover use widely available disinfectant,
 - Do not wash wedges and rings,
 - Do not iron,
 - Do not chlorinate,
 - Do not swirl,
 - Do not dry clean,
 - Do not tumble dry.
 - Protect from moisture,
 - Protect from high temperatures,
 - Dry away from sources of warmth.
- ATTENTION: Wedges and rings should be used with a cover.

REH4MAT
 MANUFACTURER OF MODERN REHABILITATION DEVICES

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BEFORE USING A PRODUCT, IT IS NECESSARY TO FAMILIARIZE YOURSELF WITH THE CONTENT OF THIS INSTRUCTION AND CHECK IF THE ELEMENTS ARE NOT WORN OUT. SAVE THE PRODUCT INFORMATION, TO BE ABLE TO REREAD THE INSTRUCTION LATER.

PAIN IS ALWAYS THE SIGN OF WARNING. IF THE PAIN MAINTAINS OR INCREASES IT IS ADVISABLE TO CONSULT YOUR DOCTOR OR PHYSICAL THERAPIST.

FOR SINGLE PATIENT USE ONLY. IF THE PRODUCT IS COVERED IN SANMED C463 IT CAN BE USED BY MANY PATIENTS, BUT ONLY AFTER DISINFECTION. USING OF THE DEVICE NOT ACCORDING TO THE MANUAL MAY CAUSE SERIOUS COMPLICATIONS.

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