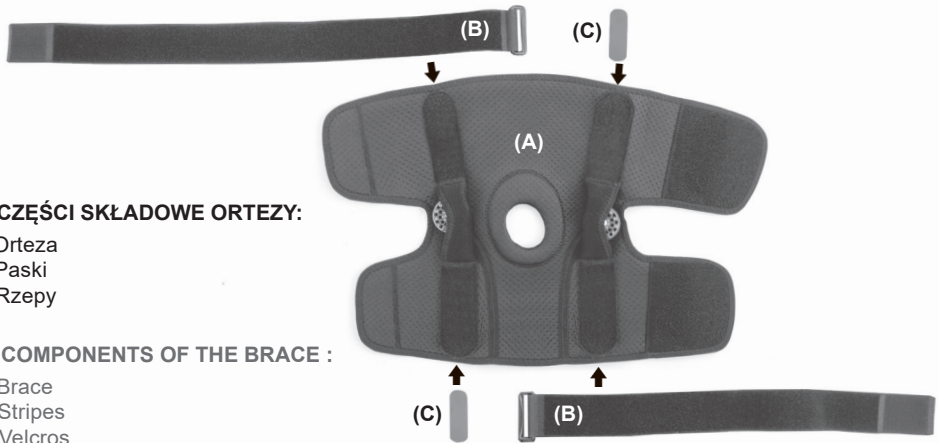
KOD KATALOGOWY	AS-KX-07
NAZWA HANDLOWA	ORTEZA KOLANA
MODEL	ORTEZA KOLANA AS-KX-07
CODE	AS-KX-07
TRADE NAME	KNEE ORTHOSIS
MODEL	KNEE ORTHOSIS AS-KX-07

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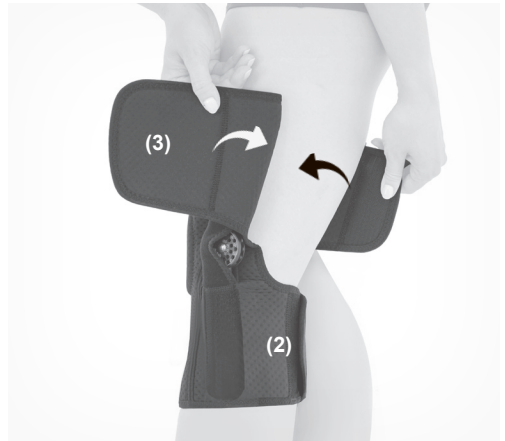
- A. Orteza
- B. Paski
- C. Rzepcy

EN: COMPONENTS OF THE BRACE :

- A. Brace
- B. Stripes
- C. Velcros

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EN: Unfasten the brace and unhook all attached parts.

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
EN: Place the orthosis (A) on the leg in such a way that the kneecap is in the peritellar opening (1) and the clocks are in the axis of the knee joint.

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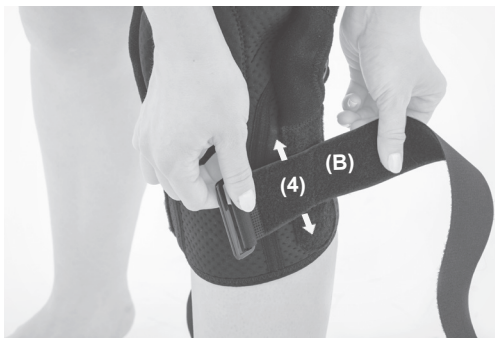EN: Fasten the brace at the back with Velcro according to the following steps (2,3).

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EN: Fasten the perimeter straps (B) to the rail channel with Velcro (4 and 5).

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EN: ATTENTION! Each of the perimeter straps can be attached at the desired height by moving up or down along the rail channel line. It is also possible to reverse the strap so that the buckle is on the outside or inside of the limb, depending on the need.

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EN: Thread the tapes (B) through the buckles and fasten with the Velcro fasteners in the order given (6,7).

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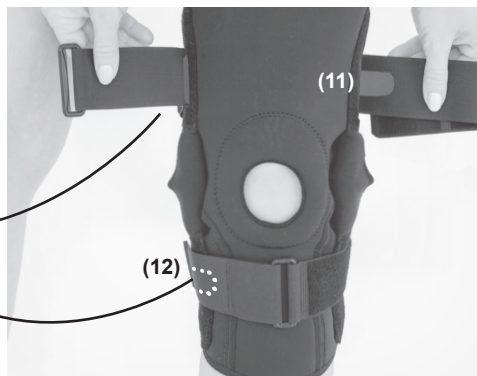
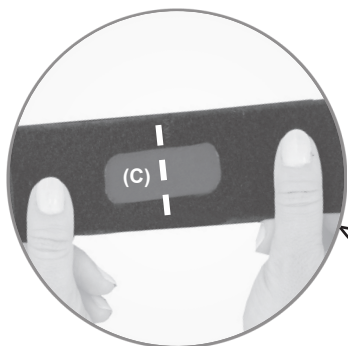
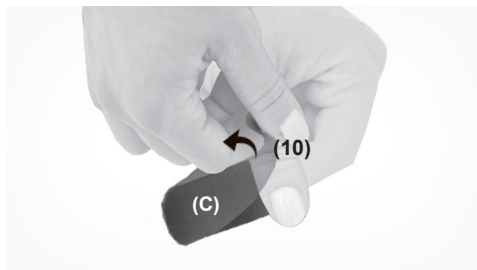
EN: On the opposite side of the buckles of the fastened orthosis, mark the places where the perimeter straps cross with the splint canals (8,9).

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EN: Peel off the protective foil (10) from the Velcro (C).

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EN: In the places previously marked on the strips, stick the Velcro on the inside (C) and press firmly to the perimeter tapes (11,12).

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EN: Product is ready to use.