POLSKI

ENGLISH

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CUSHION

ATTENTION: BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTION COMPLETELY AND CAREFULLY AND CHECK IF PRODUCT'S ELEMENTS AND SEAMS ARE SEEDY. ONLY UNDAMAGED DEVICE AND ITS CORRECT USING ARE VITAL TO THE PROPER FUNCTIONING OF IT. SAVE THE PRODUCT INFORMATION, TO BE ABLE TO REREAD THE INSTRUCTION LATER.

PRODUCT DESCRIPTION:

Cushion is specially designed to provide user comfort. Cushion is an excellent solution for anti bedsores therapy. Reh4Mat products are innovative and are carefully made of the high quality materials. Our key objective is customer's satisfaction. In case of any questions feel free to contact with us by e-mail: biuro@reh4mat.com.



IMPORTANT:

- In case of skin lesions occurring and allergic reaction is it necessary to contact with physician.


CARE AND MAINTENANCE:

- Cushion cover should be cleaned regularly with mild detergent with a sponge or soft brush, after that it should be wiped by damp cloth and then wiped dry,
- To disinfection the cover use widely available disinfectant,
- Do not wash cushion,
- Do not iron,
- Do not chlorinate,
- Do not dry clean,
- Do not tumble dry,
- Protect from moisture,
- Protect from high temperatures,
- Dry away from sources of warmth.

ATTENTION: Cushion should be used with a cover.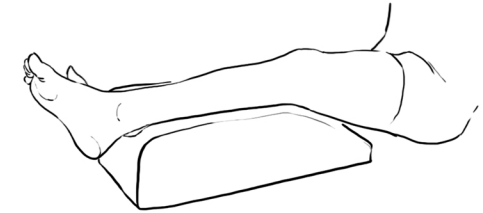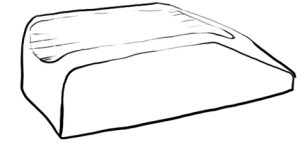

REH4MAT
 MANUFACTURER OF MODERN ORTHOPAEDIC BRACES

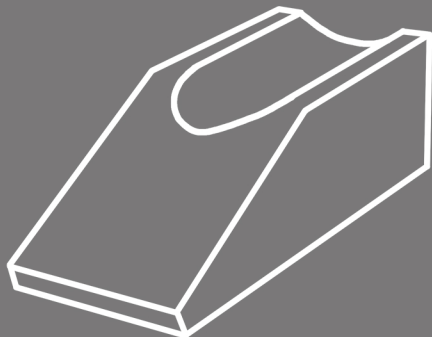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ATTENTION: USING OF THE DEVICE NOT ACCORDING TO THE MANUAL MAY CAUSE SERIOUS COMPLICATIONS.