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**INDICATION:**

Product is used to support soft tissues and bones and prevent against the bedsores in lying patients. It is used prophylactically to prevent or treat skin lesions.

Our products are appreciated by specialists (physicians, physical therapist) and patients. REH4MAT products are innovative and are carefully manufactured of the high quality materials. Our key objective is customer's satisfaction. In case of any questions feel free to contact us by e-mail: biuro@reh4mat.com.

**IMPORTANT:**

- Proper use of the product requires diagnosis by physician or physical therapist.
- Optimal mattress functioning can only be achieved by selecting the right size. The REH4MAT mattresses must be fitted by an authorized vender, physician or educated physiotherapist.
- It is necessary to control using the product, especially in children, people with memory dysfunction, psychiatric and people with sensory disturbances.
- The skin in place, where the product touches should be clean.
- Is it necessary to control the skin's condition (especially in heels and feet) and in case of skin lesions occurring is it necessary to contact with physician.
- In case of skin lesions (epidermis scrapes, scatches or eczema) using of the product is allowed after dressing applying.

**CARE AND MAINTENANCE:**

- The cotton cover can be washed in washing machine in temperature of 40°C.
  - The Sanmed C463 cover should be cleaned regularly with mild detergent with a sponge or soft brush, after that it should be wiped by damp cloth and then wiped dry. To disinfection the cover use widely available disinfectant,
  - Do not wash mattress,
  - Do not iron,
  - Do not chlorinate,
  - Do not swirl,
  - Do not dry clean,
  - Do not tumble dry,
  - Protect from moisture,
  - Protect from high temperatures,
  - Dry away from sources of warmth.
- ATTENTION: Mattress should be used with a cover.

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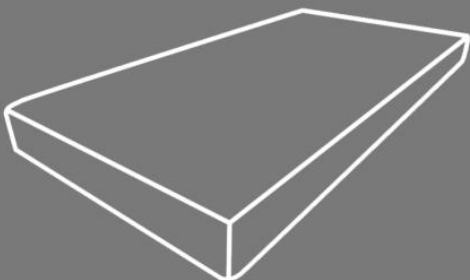
BEFORE USING A MATTRESS, IT IS NECESSARY TO FAMILIARIZE YOURSELF WITH THE CONTENT OF THIS INSTRUCTION AND CHECK IF THE ELEMENTS ARE NOT WORN OUT. SAVE THE PRODUCT INFORMATION, TO BE ABLE TO REREAD THE INSTRUCTION LATER. PAIN IS ALWAYS THE SIGN OF WARNING. IF THE PAIN MAINTAINS OR INCREASES IT IS ADVISABLE TO CONSULT YOUR DOCTOR OR PHYSICAL THERAPIST.

FOR SINGLE PATIENT USE ONLY. IF THE PRODUCT IS COVERED IN SANMED C463 IT CAN BE USED BY MANY PATIENTS, BUT ONLY AFTER DISINFECTION. USING OF THE DEVICE NOT ACCORDING TO THE MANUAL MAY CAUSE SERIOUS COMPLICATIONS.

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