POLSKI

ENGLISH

DEUTSCH

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PRODUCT DESCRIPTION:

The brace is an excellent solution for effective therapy in case of injury or other conditions. Braces are appreciated by specialists (physicians, physical therapist) and patients. Reh4Mat products are innovative and are carefully manufactured of the high quality materials. Our key objective is customer's satisfaction. In case of any questions feel free to contact with us by e-mail: biuro@reh4mat.com.

IMPORTANT:

- Proper use of the brace requires diagnosis by physician or physical therapist.
- Optimal brace functioning can only be achieved by selecting the right size. The Reh4Mat brace must be fitted by an authorized vender, physician or educated psychotherapist.
- It is necessary to control using the brace, especially in children, people with memory dysfunction, psychiatric and people with sensory disturbances.
- The skin in place, where the brace touches should be clean (the creams or ointments should not be used).
- Is it necessary to control the skin's condition (especially in heels and feet) and in case of skin lesions occurring is it necessary to contact with physician.
- In case of skin lesions (epidermis scrapes, scathes or eczema) using of the braces is allowed after dressing applying.
- Chronic using the brace may cause limitation of the active and passive movement in joints. If there are no contraindications, it is recommended to exercise several times during the day. Exercises are selected individually for each patient by a master's degree or technician rehabilitation as physician has recommended.
- In case of varicose veins, Deep Vein Thrombosis (DVT), lower limb or upper limb oedema the using of brace is allowed only after contact with physician.
- Use of the brace during the night should be done only if this was prescribed by your doctor.

CARE AND MAINTENANCE:

- Wash by hands in warm water with soap in temperature of 30 C,
- Sluice carefully,
- Do not iron,
- Do not chlorinate,
- Do not dry clean,
- Do not tumble dry.
- Protect from moisture,
- Protect from high temperatures,
- Dry away from sources of warmth.

Ankle and foot orthosis stabilizes the ankle joint. It is used in case of ankle and/or foot injuries, neurological and rheumatological disorders, after cast immobilization and during physical therapy.

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FOR SINGLE PATIENT USE ONLY. USING OF THE DEVICE NOT ACCORDING TO THE MANUAL MAY CAUSE SERIOUS COMPLICATIONS.

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