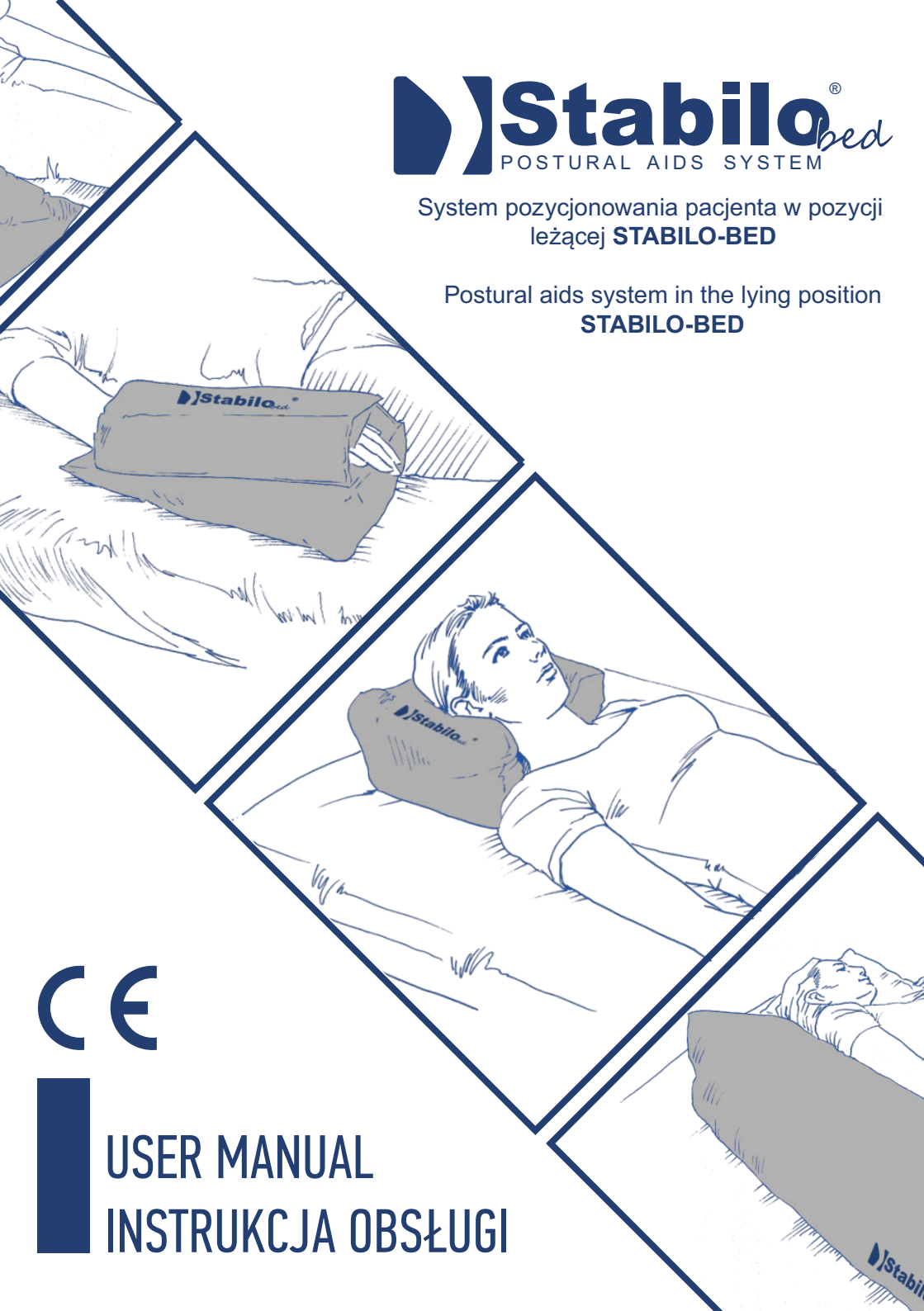
Stabilo[®]bed

POSTURAL AIDS SYSTEM

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Postural aids system in the lying position
STABILO-BED

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MODELE

EN

- Lower extremity abduction pillow **P-SS-01**
- Abduction block for knees **P-SS-03**
- Calf wedge **P-SS-06**
- Lower extremity abduction pillow **P-SS-11**
- Thermoactive lower limb raising wedge **K-SS-06**
- Anatomic-shaped cushion under head **P-SS-04**
- 'V' pillow **P-SS-15**
- Silicone pillow **P-SS-21**
- Thermoactive support cushion **P-SS-22**
- One piece trunk support **P-SS-08**
- Semi-soft bolster pillows **P-SS-14**
- Backrest with self-gripping function **K-SS-05**
- Upper extremity support with protector **P-SS-20**
- Wedge with self-gripping function **K-SS-02**
- Cylindrical cushion **P-SS-02**
- Pillow between legs **P-SS-05**
- Knee wedge **P-SS-10**
- Universal big cushion **P-SS-16**
- Thermoactive Braun splint **K-SS-07**
- Anti bedsore round cushion **P-SS-09**
- Headrest **P-SS-18**
- Neck ring cushion **P-SS-19**
- Ring cushion **P-SS-13**
- Lateral positioning wedge **P-SS-12**
- Semi-rigid bolster pillows for trunk support **P-SS-17**
- Pediatric immobilizer **P-SS-23**
- Thermoactive elevation arm wedge **K-SS-01**
- Shaped wedge **K-SS-08**
- Anti-Bedsore Donut Cushion **P-SS-24**
- Head cushion **P-SS-25**
- Orthopedic neck pillow **P-SS-26**
- Reflux Baby Wedge **P-SS-27**
- Pediatric lower limbs abduction wedge **P-SS-28**
- Universal positioning cushion **P-SS-29**
- Lower limbs positioning wedge **P-SS-30**
- Universal cushion **P-SS-31**
- **P-SS-32**
- **P-SS-33**
- **P-SS-34**
- **P-SS-35**
- **P-SS-36**
- **P-SS-37**
- **P-SS-38**
- **P-SS-39**
- **P-SS-40**
- **P-SS-41**
- **P-SS-42**
- **P-SS-43**
- **P-SS-44**
- **P-SS-45**
- **P-SS-46**
- **P-SS-47**
- **P-SS-48**
- **P-SS-49**
- **P-SS-50**
- **K-SS-09**
- **K-SS-10**
- **K-SS-11**
- **K-SS-12**
- **K-SS-13**
- **K-SS-14**
- **K-SS-15**
- **K-SS-16**
- **K-SS-17**
- **K-SS-18**
- **K-SS-19**
- **K-SS-20**
- **K-SS-21**
- **K-SS-22**
- **K-SS-23**
- **K-SS-24**
- **K-SS-25**
- **K-SS-26**
- **K-SS-27**
- **K-SS-28**
- **K-SS-29**
- **K-SS-30**
- **W-S-01**
- **W-S-02**
- **W-S-03**
- **W-S-04**
- **W-S-05**
- **W-S-06**
- **W-S-07**
- **W-S-08**
- **W-S-09**
- **W-S-10**
- **F-SS-01**
- **F-SS-02**
- **F-SS-03**
- **F-SS-04**
- **F-SS-05**
- **F-SS-06**
- **F-SS-07**
- **F-SS-08**
- **F-SS-09**
- **F-SS-10**

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STABILObed[®] is the modular 1st medical class system that is used for precise and effective positioning the patient in the lying position. **STABILObed[®]** system has a wide range of applications. The **STABILObed[®]** system consists of independent elements:

1) Different Shaped Cushions. **STABILObed[®]** system is a range of blocks and cushions, used for stabilization of different body parts. They are covered by innovative fabric **PureMed[™]**. This material is very safe for human and easy to sterilize. Particular components of system can be used for patient immobilization in diagnostic imaging (MRI, CT scan, X-ray). Depending on type, shape and purpose, **STABILObed[®]** cushions are filled with:

Granulate – micro-beads with small granulometry. The micro-beads may go smoothly inside the cushion and because of that, it can be anatomic-shaped very easy. What is more, **PureMed[™]**, with high vapour permeability, allows the air to go outside the cushion and be around the patient's body.

VISmemo[™] foam - is our innovative memory foam. The foam creates initial shaping under the first pressure. Thereafter, the foam is more soft, because of patient's temperature. **VISmemo[™]** foam is the most active in areas with high bed sores risk. Our innovative foam **VISmemo[™]** is the best solution for bedsores therapy these days.

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EN **Comfort foam** – soft foam with a lot of closed, small air bubbles. It allows to maintain the patient's shape precisely.

Off-loading foam – it's rigid technical foam, created in recycling process. The structure is closed what allows to back to initial position even after long-term lying.

Silicone – this filling consists of silicone fibers in round shape with 0,5 cm diameter. There are air inside what prevents against the sweating of the user. The cushion filled silicone is very soft and lightweight and can maintain the perfect anatomic shape. It's Pressure-resistant even after long-term lying pressure.

2)Self-adhesive elastic band sheet – The sheet is made of soft, friendly velour in white or blue color. It's base for **STABILObed**[®] cushions and wedges. All of the components of **STABILObed**[®] system are equipped with set of Velcro what prevents against its migration on mattress. The sheet is perfect for standard hospital mattresses and it's available in 3 different sizes. It's equipped with elastic band. It is possible to produce sheet in individual size according to your needs.

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INDICATIONS

Using **STABILObed[®]** postural aids system is particularly appropriate in the following cases:

- ✓ As an immobilization in diagnostic imaging (MRI, CT scan, X-ray);
- ✓ Orthopedic, neurological, rheumatologic diseases;
- ✓ Intensive patients care;
- ✓ Rehabilitation, palliative care, and positional therapy;
- ✓ Long-term care;
- ✓ Postoperative immobilization;
- ✓ Various body parts relieving;
- ✓ Bed sores prevention.

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The stabilizing and positioning **STABILObed®** system is made of innovative material called **PureMed™** in gray color. It has many advantages:

- ✓ It is very safe for patients and easy to disinfect, which makes it ideal for use in high risk of infection areas;
- ✓ There is a possibility of multiple use, it does not rub off. It is a perfect solution for hospitals or rehabilitation centers because it can be sterilized and used by numerous patients;
- ✓ It's lightweight, soft, elastic and easy to shape.
- ✓ It contains antibacterial components based on inorganic Silver Zeolite;
- ✓ It meets all the essential requirements of the medical products of different destination
- ✓ It has **Oeko-Tex® Standard 100 certification** which is given to products which have exceptionally skin-friendly parameters;
- ✓ It is used as cover for hospital mattresses, operating tables and couches;
- ✓ It is waterproof and provides high vapour permeability only in one way - the air may migrate from cushion, but the sweat, blood and urine aren't allowed into inside.

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PureMed[™] should be cleaned according to the laundry guide. Coated materials should be cleaned with the use of neutral pH soap solution (about 7) applied with a soft sponge or a damp cloth. Rub gently and wipe dry to remove the rest of the detergent. In order to keep the original colour and shine level, the procedure should be repeated every month.

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Stains should come out if you use 25% solution of ethanol. Rub gently and wipe dry to remove the rest of the detergent.

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Please, bear in mind that some stains might not be completely removed from the cleaned surface. It depends on several factors like active dyes permanently migrating to the material surface, the material composition and finish, or chemical composition of dirt as well as the time the soiling has remained on the material surface.

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You should pay particular attention to the generally available domestic cleansers whose components might react with the soiling and cause an effect contrary to what was originally intended.

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Remember that **PureMed**[™] should never be cleaned with domestic cleansers and cleaning agents that are generally recommended for 'eco-leather'.

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The following table shows the methods and agents approved for disinfection **PureMed™**.

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- **PureMed™** fabric can be disinfected and steam sterilized at 121°C for 20 minutes, minimum 50 cycles.
- The fabric can also be disinfected with hydrogen peroxide, including hydrogen peroxide gas.
- The deterioration of the visual and aesthetic properties (such as wrinkling of the surface) does not affect the fabric's features (both physical and mechanical)

The product is not covered by the guarantee when:

- The damage is caused by high temperature, caustic fluids or fire;
- The permanent discoloration caused by contact with clothing occurs (such as leather, jeans, suede, corduroy, etc.);
- It has traces from a pen, ink, markers, etc.;
- There is a mechanical damage caused by pets or other users;
- The defects are caused by improper maintenance.

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EN

PureMED™ COVER CARE GUIDE*

Wash at 95°C or less. Gentle process (min 50 cycles)

Tumble dry. Medium temperature

Non-Chlorine

Do not iron

Do not dry clean

B-SS SELF-ADHESIVE SHEET CARE GUIDE

Wash gently at or below 40°C.

Do not iron

Non-Chlorine

Do not tumble dry

Dry flat. Keep away from heat sources.

THE TERRY COVER CARE GUIDE

Wash at or below 40°C.

Iron inside out, low temperature.

Non-Chlorine

Do not dry clean

* The care guide relates to the PureMed™ cover only.

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SAFETY RULES

1. Before using our products, there is necessary to sterilize them with allowed agents shown in table 2 (p. 10) every time.
2. There is recommended to use sheet by single user. In case of using it by more than 1 person, there is necessary to wash it after using, according to maintain rules (p. 11).
3. For proper function, the sheet have to cover the mattress precisely. It cannot be fold. It should be tight and slid under the mattress.
3. The service valve should be always closed. You can only open it to adjust the granulate volume.
4. Keep our products away from sharp tools and any other items, that can damage **STABILObed[®]** products.
5. In case of mechanical damage one of our product, there is necessary to return it to manufacturer to repair.
6. **STABILObed[®]** system should be kept in dry and clean room. Keeping the other items on our products is forbidden.

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EN

ATTENTION!

Over time, the polystyrene granules, which fill the **STABILObed[®]** cushions, may give signs of flattening. As a result, the volume of the cushions will decrease, which is their natural feature. If this situation arises, we recommend replacing or refilling the granules in the cushion. It can be easily done with the use of a special funnel which is in the cushion and a granule bag, which is available in two sizes.

The decreasing volume of the cushions is their natural feature and it shall not give rise to a complaint.

ATTENTION!

Before using the **STABILObed[®]** products, always make sure that the place of use (e.g. bed, mattress, seat) is free from sharp edges, metal parts, springs and other elements which may damage the product. The manufacturer is not liable for the mechanical damage of the product!

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HOW TO FILL THE CUSHION INSERT WITH GRANULES?

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To supplement the granulaes bag with **STABILObed[®]** granules:

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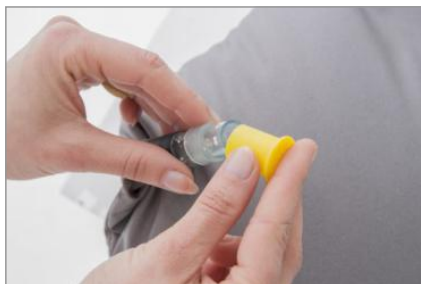
5. Place the funnel with the granulate inside the pocket and pour in the desired amount of filling.

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6. After filling, put the protection cap on the granule bag, close the cotton insert and place it in the **PureMed™** cover (perform steps 1-4 in reverse).

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