


INSTRUKCJA REGULACJI KĄTA ZGIĘCIA I WYPROSTU W SZYNI DWUOSIOWEJ 2R Z AMORTYZACJĄ WYPROSTU


Orteza została wyposażona w szynę boczną, która posiada **blokadę kąta zgięcia (A)** w zakresie: 0°, 10°, 30°, 50°, 70°, 90°, 110° oraz **blokadę wyprost (B)** w zakresie: 30°, 50°, 70°, 90°, 110°.




rys.1

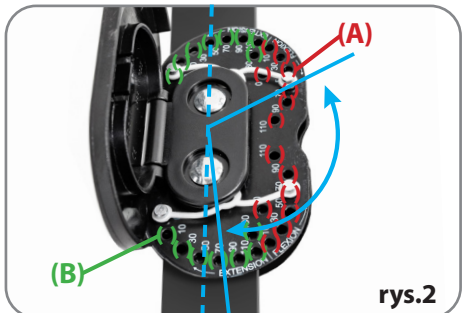

rys.2

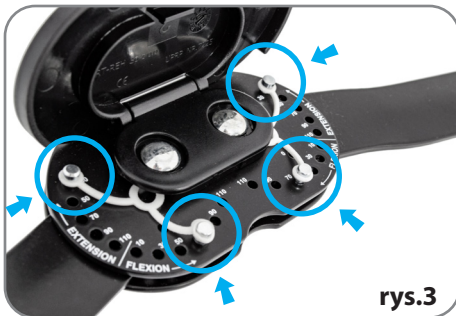

rys.3

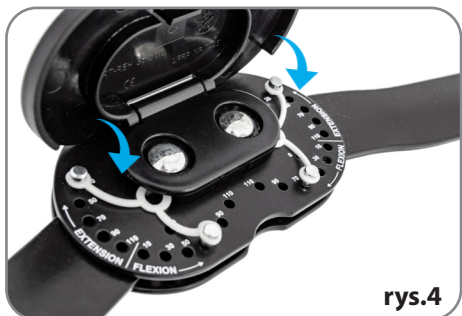

rys.4

UWAGA: Przed ustawieniem szyny, usuń wszystkie blokujące nity.




1. Otwórz klawkę zabezpieczającą (**rys.1**)
2. Ustaw ortezę w przybliżonym położeniu, w którym chcemy ograniczyć **zgięcie/wyprost** (**rys.2**)
3. Zablokuj ramiona szyny (dwustronnie) wkładając we właściwe otwory 4 nity blokujące (**rys.3**)
4. Zamknij klawkę (**rys.4**) i sprawdź poprawność działania i zakres ruchów zegara

ZAKRES RUCHÓW I JEGO WARTOŚCI KĄTOWE USTALA LEKARZ PROWADZĄCY!

UWAGA: Jeżeli orteza wyposażona jest w dwie szyny boczne, regulacji należy dokonać osobno na obydwu szynach bocznych, ustawiając ten sam kąt zgięcia lub wyprost (punkt 1- 4). Szyny boczne nie wymagają konserwacji.

ROM ADJUSTMENT INSTRUCTION OF 2-AXIAL SPLINT 2R WITH EXTENSION ABSORBING

The brace is equipped with a side splint with a **bending flexion (A)** in the scope of: 0°, 10°, 30°, 50°, 70°, 90°, 110° and **the extension lock (B)**: 30°, 50°, 70°, 90°, 110°.

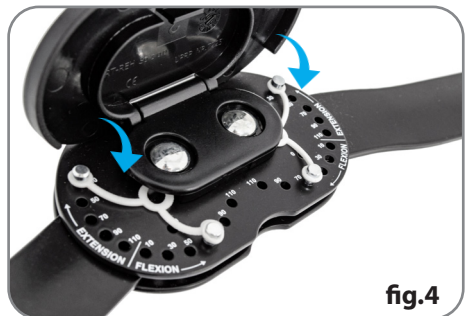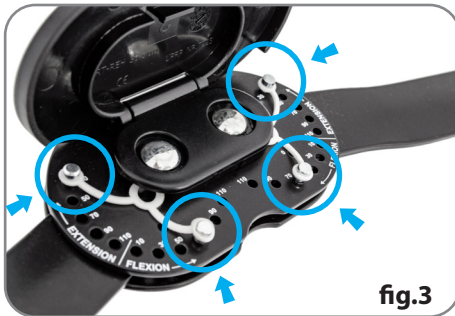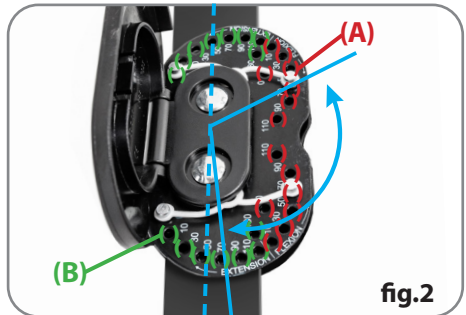

ATTENTION: Before setting up the splint, remove the screws pins from it.

1. Open the security flap (**fig.1**)
2. Position the brace in the approximate position where we want to limit the **flexion/extension** (**fig.2**)
3. Lock the rail arms (on both sides) by inserting 4 locking rivets into the appropriate holes (**fig.3**)
4. Close the flap (**fig.4**) and check the correctness of operation and the range of clock movements

THE ANGLE OF FLEXION AND EXTENSION IS SPECIFIED BY THE DOCTOR OR PHYSIOTHERAPIST!

ATTENTION: If the brace is equipped with 2 slide splints, adjustment is done on each side splint separately, setting the same angle of flexion and extension on both splints (point 1-4). Side splints do not require maintenance.



REH4MAT

Ul. Piaski 47, 36-060 Głogów Małopolski | Polska

tel. +48 16 62 142 20

Fax +48 16 62 21 42 13

e-mail: biuro@reh4mat.com

www.probraces.com

www.reh4mat.com