PL

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## SPLINT WATER BATH

### INSTRUCTIONS FOR USE

Please read the document before putting the device into operation. Failure to follow these instructions may result in damage to the equipment, loss of warranty, and / or personal injury.

When using this product for the first time, check that the external power supply meets the product's power requirements, i.e. 230V AC. The power supply must always be used in accordance with the power requirements.

The power socket must be connected to the ground wire. When in use, do not leave the device unattended for a long time, and if necessary, turn off the power. If there is a power failure, also turn off the power.

A thermostatic tank of the water bath is made of a high quality stainless steel using precise technology, that makes it resistance to temperature and corrosion.

### GENERAL INFORMATION

The device should be used in hospitals, clinics and other medical facilities to heat and soften the thermoplastic material used in the process of providing the patient with a device that immobilizes a given part of the body. For indoor use only.

### TECHNICAL PARAMETERS

- Inner tank dimensions: 680\*500\*120mm
- Outer tank dimensions: 825\*565\*165 mm
- Voltage: 230 V, 50 Hz
- Power: 1800 W
- Temperature range: from room temperature to max. 100 °C

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- (1) **Display:** displays the measurement temperature or the set switching temperature, as well as various symbols / prompts,
- (2) **OUT (heating indicator light):** lights up when the bath becomes heating;
- (3) **ALM (alarm indicator lamp):** lights when there is an alarm,
- (4) **RUN / AT (run or auto-tune):** lights up during bathing operation, blinks in auto-tune mode,
- (5) **Function key:** used to set parameters and confirm parameter modification,
- (6) **Shift button:** used to move left and right when modifying parameters,
- (7) **Decrease button:** used to adjust a parameter or stop heating,
- (8) **Increase button:** used to adjust a parameter or start auto-tuning.

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## II. Main settings

### ▪ Temperature settings

1. Press the **function key**  to set the heating temperature.
2. Using the **Shift button** , we can change the range of values we want to set.
3. The temperature value is increased with the **increase button**  or decreased with the **decrease button** .
4. In order to accept the set temperature, press the **function key**  again, then the display will show the actual temperature of the water in the bathtub and start heating the water.
5. To turn off the heating, hold down the **button**  for 5 seconds, then the message „END” will be displayed and the bath will stop heating. To turn the device back on, press the **button**  again for 5 seconds.
  - If the bathtub temperature fluctuates greatly, press and hold the **button**  for 5 seconds while heating. The device enters the auto-tuning state, the **RUN/AT** indicator flashes, and the ideal parameters are set. The temperature will be controlled according to the new parameters.
  - If the display shows **0000**, it means there is a short circuit or the temperature is outside the measuring range.
  - Boiling function: When the **temperature is set = 99.9 ° C**, the heating is on maximum for the water to boil.

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**EN**

### III. Maintenance

1. The device must be used in accordance to this manual. It is not allowed to use it for other purposes than for heating thermoplastic material.
2. In the case of frequent or long-term use of the device at high temperatures, regularly check the controls for drops of water, dirt, etc. Check the water quality and follow the cleaning procedure of the water tank, which will extend the life of the device.
3. Disconnect the device from the power supply to perform maintenance or repair.

**EN**

### IV. Cleaning instruction

- Before cleaning the device, turn off the power and disconnect the device from the power source.
- Before cleaning, make sure that the device is cool and the heaters are not warm in order to avoid burns.
- Using the tap on the side of the device, drain the water from the tank. Then, before starting cleaning, make sure that all the water has been drained.
- Open the cover of the tank, then pull out the grate on which we put the thermoplastics. Use a clean, soft and absorbent cloth to clean the water tank. Do not use the same cloth to clean the cover to prevent scratching and rusting.
- Clean the bathtub with soap or mild detergent and warm water. Avoid cleaning agents containing chloride or salt. A stainless steel cleaner can be used to increase its performance and lifespan.
- When cleaning, be careful with the heater and temperature sensor, so as not to damage them.
- Do not scrub the tank or the cover of the device too hard to prevent it from creating numerous scratches.
- After cleaning, the device should be put in a clean and ventilated place in order to be completely dry.
- To work again, refill the bathtub with cold water to the level of 2-3 cm above the plastic grate.

#### ATTENTION!

The water should be changed at least once a week and whenever it looks cloudy.

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## V. Additional information

- Equipment should be properly grounded to avoid accidents.
- Keep the device away from children.
- The device should be operated on a flat table in a dry, clean, ventilated place. Do not move the machine while it is running.
- To avoid personal injury, do not touch the tank cover during operation.
- Do not put your hands inside the tank to fish out. Use special gripping pliers to prevent burns.
- It is strictly forbidden to heat the device without water. This can damage the device.
- After each use, the internal and external surfaces of the device must be cleaned. If there are water stains, wipe them off with a cloth. Do not use chemical thinners, gasoline, etc. for cleaning. The natural wear of the device is normal after long-term use.
- Do not use the device near gases or flammable liquids, as a fire or explosion may result.
- Chlorine should not be used as it can damage the surface of the stainless steel.
- Allow the device to cool down completely before emptying or moving the device.
- Do not use in direct sunlight.