BodyMap
VACUUM SYSTEM BY ORTHOMAN

KATALOG PRODUKTÓW
PRODUCT CATALOGUE

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BodyMap® System refers to first class medical products which are designed for adults as well as children with cerebral palsy. All these products provide perfect stabilization during seating or lying position. What is more, it is a perfect choice for the disabled due to the fact that it prevents bed sores and muscle stiffness.

BodyMap® System uses vacuum method. This method refers to pillows and mattresses which are filled with polystyrene granules thanks to which products can model according to patient's body. Polystyrene granules move freely inside the device thanks to which it is possible to model the cushion. The cushion perfectly models according to patient's body when patient either sits or leans on the cushion. The shape of cushion doesn't change easily because of the fact that the air is sucked out of the cushion.

Our system is designed with the intention of maintaining correct sitting. Cushions, backrests and pads make it possible to have correct support for trunk, hips, head and upper as well as lower limbs.

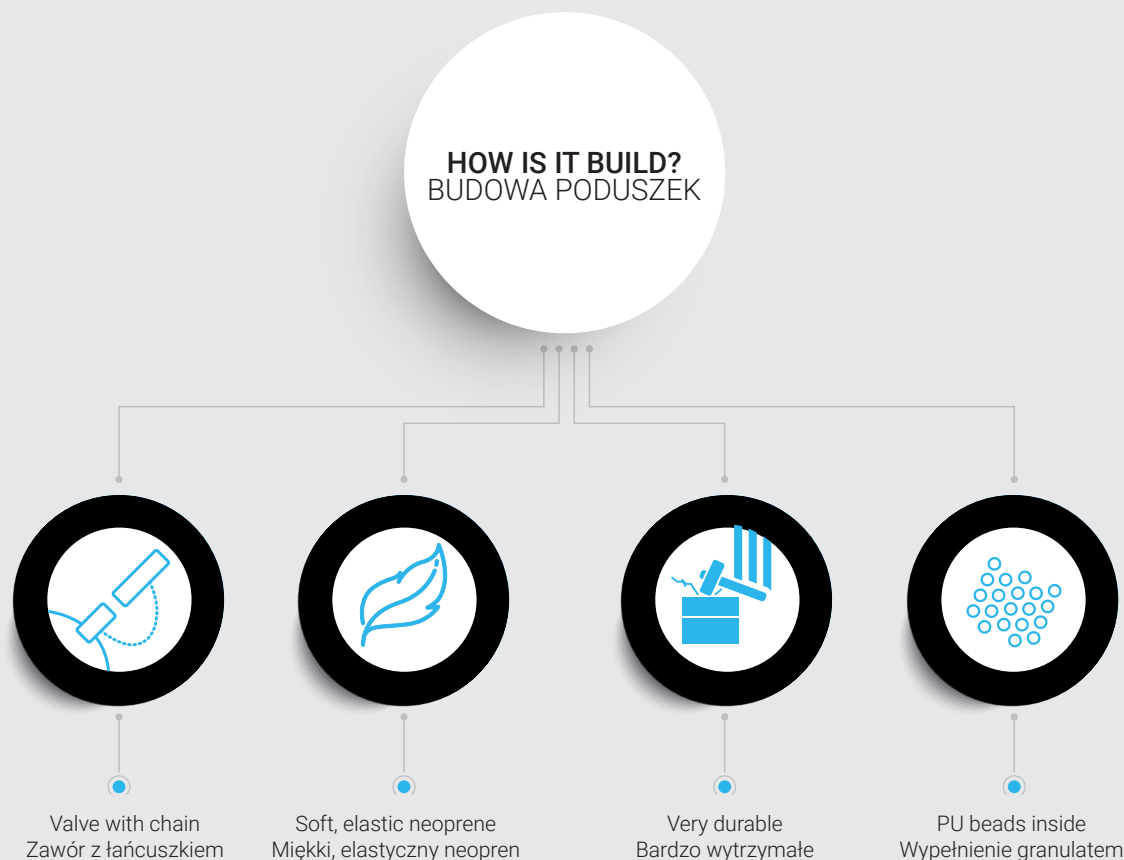
BodyMap® System is perfect for patients with cerebral palsy, scoliosis or hump. The product is shaped according to patient's body shape thanks to which unit pressure is diminished. It has a significant influence on bed sores prophylaxis. In addition, it is possible to determine the level of product's hardness with the aid of vacuum method.

BodyMap® products are made of soft, elastic neoprene foam and they perfectly adjust to patient's body. The use of neoprene foam has a significant influence on durability of the product.

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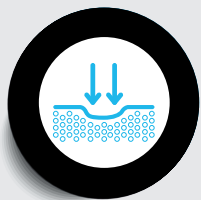
strony
06 - 23

BODYMAP® CUSHIONS
PODUSZKI BODYMAP®

BodyMap® A is a seat cushion which helps to stabilize and relieve patients as well as makes it possible to support hips in a wheelchair, positioning chair or ordinary chair. What is more, it is also possible to model an abduction wedge, raise one hip in order to maintain the correct position of spine or off-load sensitive body parts. The shape of the seat cushion can be easily changed if necessary.

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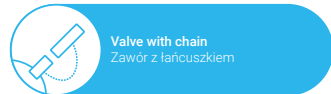
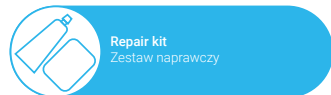
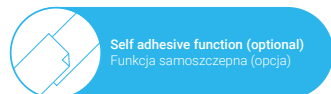
Included / W zestawie:


Key features

- Shaped form last for a long time
- Easy to give a demanded shape
- Possibility of mounting the cushion in a wheelchair, positioning chair, chair or in a car seat
- Optional cover made of VISmemo™ thermoactive foam
- Self-adhesive function (optional)
- Waterproof

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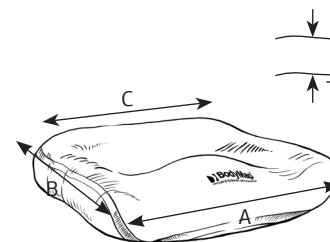BodyMap® A+ – a sitting cushion with high lateral supports and a thigh abduction wedge. Our cushion helps keep a proper sitting position. A special abduction wedge reduces contractures of adductors and maintains the legs in the required position. An anatomic shape and lateral supports of the **BodyMap® A+** cushion prevent the anterior pelvic tilt which provides comfort for bony prominences such as coccyx and sitting bones. This solution reduces the risk of bedsores.

Our **BodyMap® A+** cushion is the great product for people with neurological conditions and as an orthopedic prophylaxis after e.g. hip replacement.

Indications

- Bedsores
- Contracture of the adductor muscles of the hip
- Neurogenic disease with increased muscle tension within the pelvis
- Hip replacement

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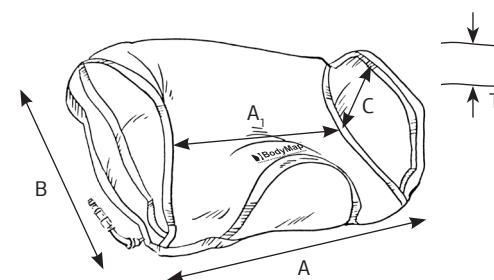
Key features

- Shaped form last for a long time
- Easy to individual shape modeling
- Use in wheelchair, positioning chair, chair or in the car
- Optional cover made of thermoactive VISmemo™ foam
- Self adhesive function (optional)
- Waterproof

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BodyMap® B is a back cushion which helps to stabilize and relieve patients. It provides comfort and correction of scoliosis and back curvature, at the same time improving the poor posture. The cushion increases the comfort of the sitting patient by ensuring the correct posture and the body stabilization. It also allows to obtain a correct and stable posture by patients with a hump. The shape of our cushion can be easily changed if necessary.

Indications

- Weakness or limpness of paraspinal muscles, the trunk and abdominal muscles
- Back and neck bedsores prophaxis
- Discopathy
- "Round back" (kyphotic spine)
- Flat back
- Flat back
- Cerebral palsy
- Poor posture
- Excessive muscles tension
- Scoliosis
- Myotonia

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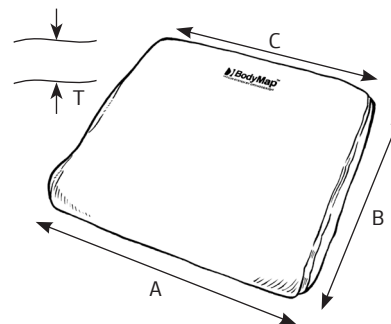
Key features

- Shaped form lasts for a long time
- Easy to give a demanded shape
- Helps to keep a proper position
- Possibility of mounting the cushion in a wheelchair, positioning chair, chair or in a car seat
- Optional cover made of VISmemo™ thermoactive foam
- Self-adhesive function (optional)
- Waterproof

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BodyMap® B+ backrest cushion with lateral supports is highly recommended for people with back and trunk muscle weakness and for everybody who's in a sitting position for a long time, e.g. in a wheelchair. The cushion positions the patient and reduces the pressure on the bone eminence, i.e. shoulder blades or vertebrae. Our positioning cushion also reduces excessive stress on skin tissue which limits the risk of bedsores.

Indications

- Weakness or limpness of paraspinal muscles, the trunk and abdominal muscles
- Back and neck bedsores pro-phaxis
- Discopathy
- "Round back" (kyphotic spine)

- Flat back
- Cerebral palsy
- Poor posture
- Excessive muscles tension
- Scoliosis
- Myotonia

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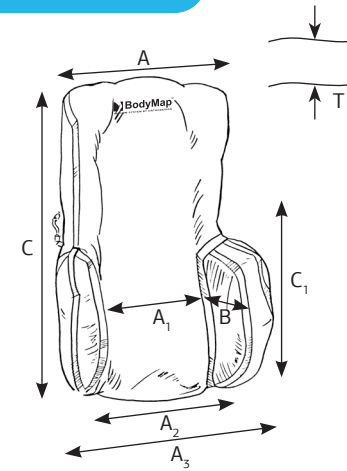
BodyMap® C is a vacuum cushion with lateral supports and a headrest, which provides comfort and correction of scoliosis and back curvature, at the same time improving the poor posture. The cushion increases the comfort of the sitting patient by ensuring the correct posture and the body stabilization. It also allows to obtain a correct and stable posture by patients with a hump. **BodyMap®** products are ideal for children with cerebral palsy and patients with muscle weakness or spasticity. Wide lateral supports provide perfect stabilization of the patient whereas the headrest is useful while adjusting the correct position of the patient's head. The shape of the cushion can be easily changed if necessary.

Indications

- Weakness or limpness of paraspinal muscles, the trunk and abdominal muscles
- Back and neck bedsores prophaxis
- Discopathy
- "Round back" (kyphotic spine)
- Flat back
- Cerebral palsy
- Excessive muscles tension
- Scoliosis
- Myotonia

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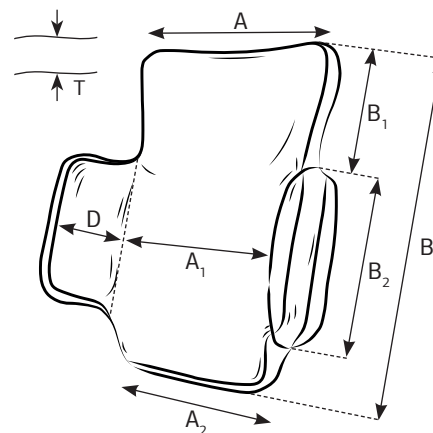
BodyMap® AB is a vacuum chair which functions both as a backrest and seat. It is useful while transportation of patients in cars and wheelchairs. The cushion makes it possible to support hips and provides comfort and correction of scoliosis and back curvature, at the same time improving the poor posture. It also allows to obtain a correct and stable posture by patients with a hump. The shape of the cushion can be easily changed if necessary.

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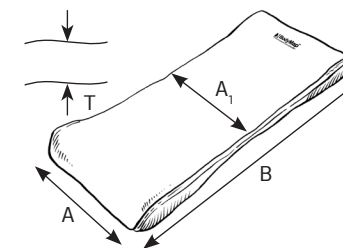
Key features

- Shaped form lasts for a long time
- Easy to give a demanded shape
- Self-adhesive function (optional)
- Helps to keep a proper position
- Can be used as a positioning chair
- Possibility of mounting the cushion in a wheelchair, positioning chair, chair or in a car seat
- Optional cover made of VISmemo™ thermoactive foam
- Waterproof

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BodyMap® AC is a vacuum chair with lateral supports and a headrest which functions both as a backrest and seat. It is useful while transportation of patients in cars and wheelchairs. The cushion makes it possible to support hips and provides comfort and correction of scoliosis and back curvature, at the same time improving the poor posture. It also allows to obtain a correct and stable posture by patients with a hump. Wide lateral supports provide perfect stabilization of the patient whereas the headrest is useful while adjusting the correct position of the patient's head. The shape of the cushion can be easily changed if necessary.

Indications

- Cerebral palsy
- Scoliosis and back curvature
- Bedsores prophylaxis
- Helps to keep the head in correct position
- Excessive muscles tension (hypersthenia)
- Myotonia

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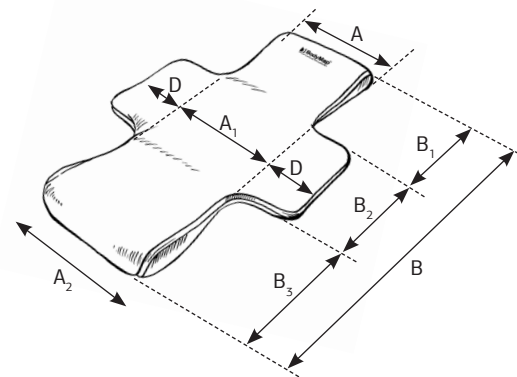
Key features

- Helps to transport patients in vehicles, wheelchairs, etc.
- Shaped form lasts for a long time
- Easy to give a demanded shape
- Self-adhesive function (optional)
- Helps to keep a proper position
- Can be used as a positioning chair
- Optional cover made of VISmemo™ thermoactive foam
- Waterproof

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BodyMap® D is an orthopedic headrest which can be easily shaped to the patient's head. It can be mounted on most of the wheelchairs, positioning chairs as well as standing frames. The shape of headrest can be easily changed if necessary. One side of the **BodyMap® D** headrest is laminated with high quality elastic polyester whereas the second one is laminated with a self-adhesive fabric which makes it possible to attach various elements with the use of Velcro to it. Thanks to that function, stabilizing the patient's head or mounting the headrest in any place, e.g. in a stroller, chair or car seat, is now possible. Additionally, the self-adhesive function prevents the headrest from moving.

Indications

- Bedsores
- Contracture of the adductor muscles of the hip
- Neurogenic disease with increased muscle tension within the pelvis
- Hip replacement

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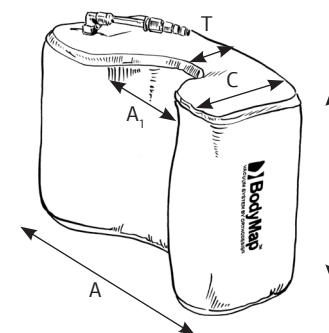
Key features

- Shaped form lasts for a long time
- Easy to give a demanded shape
- Possibility of mounting the headrest in a wheelchair, positioning chair, special needs stroller
- Optional cover made of VISmemo™ thermoactive foam
- Self-adhesive function
- Waterproof

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A butterfly-shaped orthopedic headrest for head and neck support which can be easily shaped to the patient's head. It can be mounted on most of the wheelchairs, positioning chairs as well as standing frames.

One side of the **BodyMap® DX** headrest is laminated with high quality elastic polyester whereas the second one is laminated with a self-adhesive fabric which makes it possible to attach various elements with the use of Velcro to it. Thanks to that function, stabilizing the patient's head or mounting the headrest in any place, e.g. in a stroller, chair or car seat, is now possible.

Additionally, the self-adhesive function prevents the headrest from moving.

Indications

- Cerebral palsy
- Bedsore prophylaxis
- Helps to keep the head in the correct position
- Myotonia

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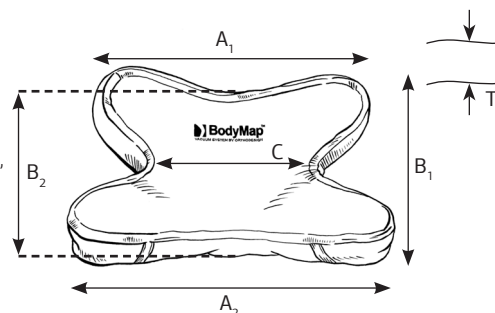
BodyMap® DY is a straight orthopedic headrest which can be easily shaped to the patient's head. It can be mounted on most of the wheelchairs, positioning chairs as well as standing frames.

One side of the **BodyMap® DY** headrest is laminated with high quality elastic polyester whereas the second one is laminated with a self-adhesive fabric which makes it possible to attach various elements with the use of Velcro to it. Thanks to that function, stabilizing the patient's head or mounting the headrest in any place, e.g. in a stroller, chair or car seat, is now possible.

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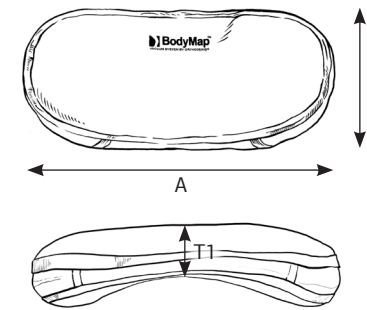
BodyMap® DZ is a triangular orthopedic headrest for head and neck support which can be easily shaped to the patient's head. It can be mounted on most of the wheelchairs, positioning chairs as well as standing frames.

One side of the **BodyMap® DZ** headrest is laminated with high quality elastic polyester whereas the second one is laminated with a self-adhesive fabric which makes it possible to attach various elements with the use of Velcro to it. Thanks to that function, stabilizing the patient's head or mounting the headrest in any place, e.g. in a stroller, chair or car seat, is now possible. Additionally, the self-adhesive function prevents the headrest from moving.

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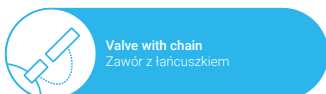
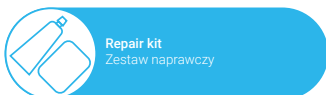
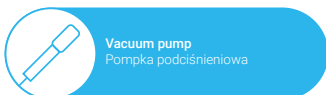
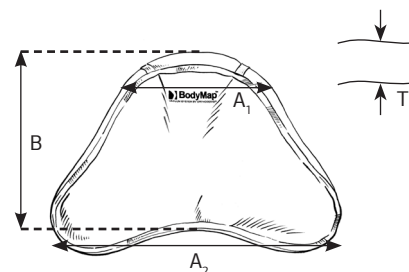
BodyMap® E is a special upper limb support for wheelchairs, positioning chairs and standing frames which can be easily shaped to the patient's arm. It can be mounted on most of the wheelchairs, positioning chairs as well as standing frames. The shape of the arm support can be easily changed if necessary.

One side of the **BodyMap® E** arm support is laminated with high quality elastic polyester whereas the second one is laminated with a self-adhesive fabric which makes it possible to attach various elements with the use of Velcro to it. Thanks to that function, stabilizing the patient's arm or mounting the support in any place, e.g. in a stroller or chair, is now possible. Additionally, the self-adhesive function prevents the arm support from moving.

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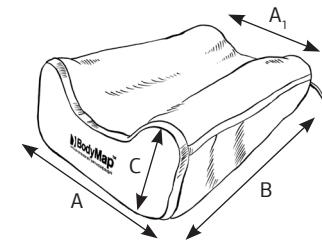
Key features

- Shaped form lasts for a long time
- Easy to give a demanded shape for better rehabilitation
- Possibility of mounting the headrest in a wheelchair, positioning chair, special needs stroller
- Optional cover made of VISmemo™ thermoactive foam
- Self-adhesive function
- Waterproof

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One-sided trunk stabilizer which can be easily shaped to patient's body. They are perfect for bed sores prevention. What is more, they can be used as post-operative supply where it is necessary to immobilize the patient in side-lying position. The shape of trunk support can be easily changed if necessary. One side of **BodyMap® H** trunk support is laminated by elastic polyester and the second one is laminated with the self-adhesive fabric that allows to attach additional tapes, straps as well as pelottes which stabilize patient's trunk. Self-adhesive function makes it easier to assembly trunk support. If the product is placed in hospital bed, it is necessary to use special bed sheet produced by **REH4MAT®**. This bed sheet has self-adhesive function.

Indications

- Cerebral palsy
- Bedsore prophylaxis
- As an immobilization on side position

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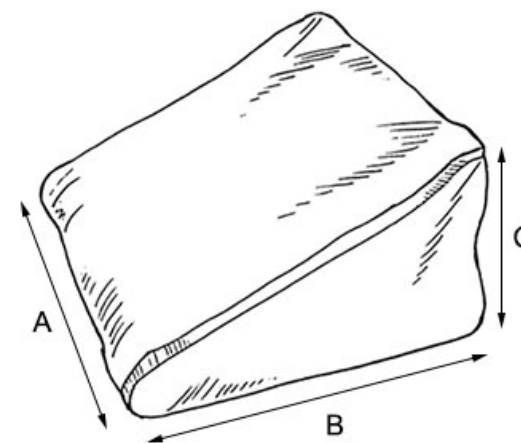
BodyMap® K is a stabilizing mattresses which can be easily shaped to the patient's body. Due to that quality, the **BodyMap®** products support every part of the patient's body and therefore are ideal in the bedsore prophylaxis. Using them guarantees the correct positioning of the trunk, pelvis, upper and lower limbs and the head. With the **BodyMap®** products, the posture defects can be easily corrected. The **BodyMap® K** mattress is laminated with high quality elastic polyester and is covered with a self-adhesive fabric at the sides. As a result, various elements can be attached to it with the use of Velcro. This allows you to stabilize the patient on the mattress.

Indications

- Cerebral palsy
- Bedsore prophylaxis
- Poor position of trunk, pelvis, upper and lower limbs and the head
- Poor posture
- Excessive muscles tension (hypersthenia)

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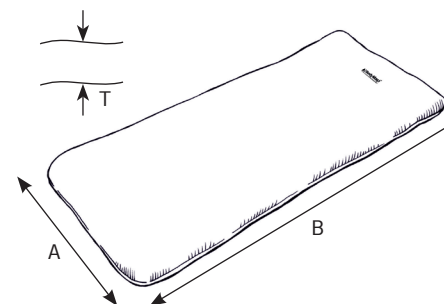
Key features

- Shaped form lasts for a long time
- Easy to give a demanded shape
- Optional cover made of VISmemo™ thermoactive foam
- Self-adhesive function (at the sides)
- Waterproof

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BodyMap® M cushion is an excellent product in addition to a daily rehabilitation. If your patient uses our standing frame, wheelchair or positioning chair with a table, you can mount our cushion on it and continue the therapy!
 You can shape the cushion according to the patient's needs, so you can place and maintain the hands and forearms in the required position.
 Our cushion precisely corrects upper limb deformations.

Indications

- Deformations of the hands and forearms (neurogenic, rheumatoid, muscle and joint contractures)
- Bedsore prophylaxis in the forearm, wrist and hand area
- Paresthesia resulting from intra-articular compression of the ulnar nerve (e.g. in RA)
- Painful rheumatoid nodules

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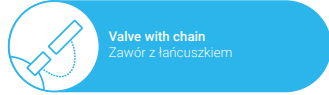
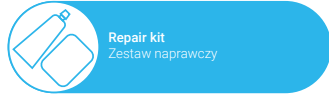
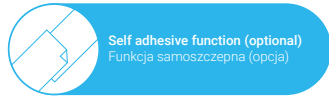
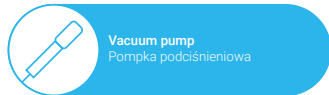
Included / W zestawie:


Key features

- Helps to put the hands in the required therapeutic position
- Corrects deformations and distortions in the hands and wrists area
- Shaped form lasts for a long time
- Easy to give a demanded shape
- Helps to keep a proper position
- Optional cover made of VISmemo™ thermoactive foam
- Self-adhesive function (optional)
- Waterproof

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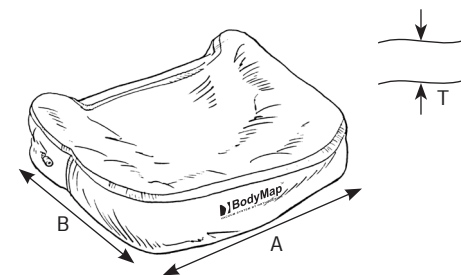BodyMap® N is a feet correction cushion which helps to improve the position of adducted or abducted feet while sitting or standing. The cushion keeps deformed feet in the required therapeutic position. It is a great product for passive rehabilitation of club feet and other deformations resulting from neurological or rheumatic conditions.

The **BodyMap® N** cushion can be used both in a sitting and standing position, e.g. during doing the homework in our standing frame.

Indications

- Bedsore prophylaxis in the feet and ankles area (mainly calcaneus)
- Ankle joint deformations (e.g. clubfoot)
- Neurogenic diseases with increased or impaired muscle tension
- Hallux valgus
- Rheumatoid foot

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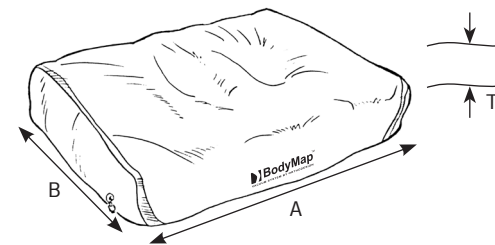
Key features

- Helps to put the feet in the required therapeutic position
- Corrects deformations and distortions in the feet and ankle joints area
- Shaped form lasts for a long time
- Easy to give a demanded shape
- Helps to keep a proper position
- Optional cover made of VISmemo™ thermoactive foam
- Self-adhesive function (optional)
- Waterproof

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BodyMap® O roller provides body stabilization and correction in the lying position. The roller keeps the patient in the required therapeutic position and corrects deformations in thy lying position. It can also be used to relieve the particular body parts. Our roller is an excellent support in the positioning therapy of lying patients with pulmonological and trophic skin disorders.

Indications

- As a postural therapy of bedridden patients
- Bedsore prophylaxis in the whole body area
- Deformations; muscular and degenerative contractures

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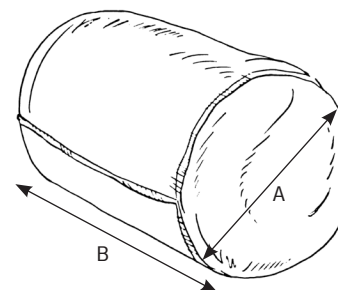
Key features

- Helps to put the patient in the required therapeutic position
- Corrects deformations and distortions
- Relieves the particular body parts of the patient
- Shaped form lasts for a long time
- Easy to give a demanded shape
- Optional cover made of VISmemo™ thermoactive foam
- Waterproof

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The bath ring **BodyMap® S Swimmer** is an excellent solution for the youngest. The bath ring supports the head, keeping it above the water line and preventing it from immersion. It helps the child do all the water rehabilitation exercise and care treatments without the risk of choking or drowning.

Special trunk straps, supplied as a standard equipment, are fastened to the cushion and keep the patient in the safe position. The patient can move the limbs and trunk freely while using the cushion which allows them to do water exercise under load.

The bath ring **BodyMap® S Swimmer** is a safe and universal product. It maintains the patient's body in a vertical or horizontal position and prevents the body rotation. The anatomic shape of the cushion provides the highest comfort of use.

BodyMap® S Swimmer is laminated with a self-adhesive fabric which makes it possible to attach various elements with the use of Velcro to it. Thanks to that function, it is easy to stabilize the patient.

Indications

- Bedsore prophylaxis in the neck area, especially at C7 height
- Neurogenic diseases with increased or weakened neck muscles tension
- Torticollis
- As a stabilization after surgeries performed in the cervical spine

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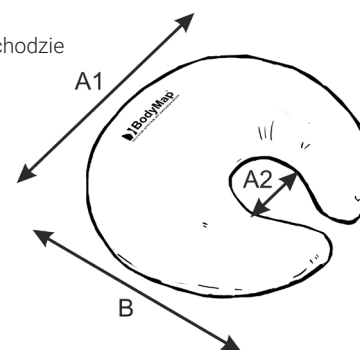
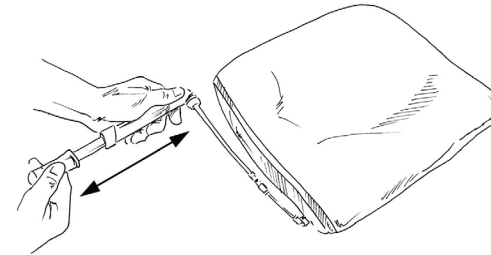
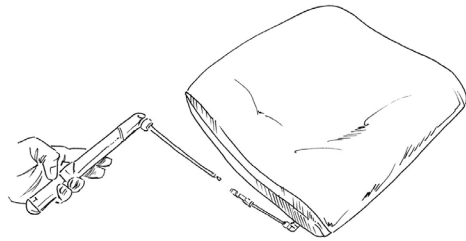
BM_001 SMALL HAND VACUUM PUMP

Small and easy to use hand vacuum pump, designed for extracting air from small and medium sized **BodyMap®** cushions.

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BM_002 LARGE HAND PUMP

Highly efficient manual pump. Ideal for mattresses and big cushions. Supplied in a set with every mattress **BodyMap® K** and sensory bean bag floor cushion **BodyMap® R/R plus**.

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ELECTRIC PUMP BM_003

Pump is ergonomically shaped and it is equipped with elastic connection pipe for BodyMap® cushions. Pipe is easy to carry and pump and, more to the point, it is comfortable to use. Pump is powered with the aid of batteries AA (1,5V x 6). Pump is efficient, average speed of pumping is 8 l/min. The maximal vacuum of pump is about -65 Kpa, thanks to which it is possible to pump enough amount of air so as to cushion is hard enough and, moreover, the shape of cushion remains the same for months.

Pump is made of solid material, namely ABS. Batteries are not included

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BM_005 ELECTRIC DUAL-FUNCTION PUMP

With electric dual-function pump it is possible to adjust the hardness of the cushion to the individual needs of the patient. This pump allows precise pumping out or in the air in order to achieve the desired hardness. Noiseless, efficient and made of durable aluminum. Strongly recommended for large cushions and mattresses.

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BM_006 AIR VACUUM BLOW GUN

The Air Vacuum Blow Gun helps to quickly increase the amount of granules in the cushion. It's especially recommended for large cushions and mattresses.

Note: The use of the gun is only possible for cushions that have a service valve (ZW), which is not standard equipment on some cushions, and is made to order.

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STABILIZING BELT WITH CLAMPS – Stabilizing belt with clamps – it is adjustable as well as non-elastic tape that is fastened with the aid of clamp. It makes it possible for a patient to have stabilization and to fasten cushion on chair, rehabilitation stroller etc.

It is possible to use parts listed below together with this product:

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STABILIZING TAPE WITH CLAMP – it is non-elastic as well as adjustable belt that is equipped with clamp and Velcro tape thanks to which tape can be fastened to self-adhesive surface of cushion. Tape can be used so as to fasten vest on cushion. Furthermore, tape together with front clasp with clamps can create pelvic belt that stabilizes the patient.

It is possible to use parts listed below together with this product:

Grommets P/2,5, P/3, P/4 and P/5

Front clasp with clamps ZPK/2,5, ZPK/3, ZPK/4, ZPK/5

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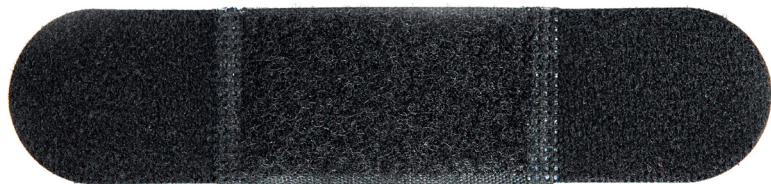
NON-ELASTIC STABILIZING BELT – it is non-elastic belt with Velcro tapes and grommet at the end. It makes it possible for a patient to have stabilization and to fasten cushion on chair, rehabilitation stroller etc. It is possible to use parts listed below together with this product:
Grommets P/4 and P/5

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GROMMETS FOR FASTENING THE TAPES – these are grommets with Velcro tapes. They can be mounted on self-adhesive surface of cushion. Grommets are used so as to adjust tapes on cushions. They have different sizes and, moreover, they can be used with various types of tapes. One set is equipped with 4 grommets.

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NON-ELASTIC CHEST BELT – it is universal and adjustable belt that is used so as to stabilize the patient. Moreover, it makes it possible to fasten **BodyMap®** cushion on chair, rehabilitation stroller etc. Belt is equipped with fastener made of Velcro tapes thanks to which it can be adjusted at any length. Additionally, belt is secured by means of tape with clamp.

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FASTENING TAPE FOR HEADREST – it is non-elastic tape that can be fastened to headrest with the aid of endings with Velcro tapes. Tape is used so as to stabilize patient's head in **BodyMap®** headrest. It is possible to use parts listed below together with this product:
Fastener with rectangular grommet SP/2,5 and SP/5

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NON-ELASTIC STABILIZING TAPE WITH CLAMP – it is non-elastic as well as adjustable belt with clamps at the ends. Tape can be used together with front clasp with clamps thanks to which pelvic belt is created and stabilization of the patient is possible. Furthermore, it is possible to fasten cushion on chair, rehabilitation stroller etc. Tape can be also used so as to fasten vests on cushion.

It is possible to use parts listed below together with this product:

Grommets P/2,5, P/3, P/4 and P/5

Front clasp with clamps ZPK/2,5, ZPK/3, ZPK/4, ZPK/5

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FRONT CLASP WITH CLAMPS – this part is equipped with clamps thanks to which it is possible to use it together with one-sided tape with clamp. Pelvic belt that stabilizes the patient can be fastened to **BodyMap®** cushion with the aid of Velcro tapes and self-adhesive surface of **BodyMap®** cushion.

It is possible to use parts listed below together with this product:

One-sided tape with clamp TJK/63/2,5, TJK/63/3, TJK/63/4, TJK/63/5

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FASTENING BELT FOR HEADREST – it is non-elastic belt that is used so as to fasten **BodyMap®** headrest on stroller's frame, chairs etc. **BodyMap®** headrest has self-adhesive surface thanks to which belt with Velcro tapes can be fastened to cushion and, moreover, headrest can be mounted to frame at any height.

It is possible to use parts listed below together with this product:

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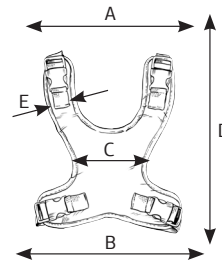
4 POINTS SAFETY VEST – is a vest stabilizing the trunk. It has a wide adjustment range, which makes it easier to fit it to the user. The vest is made of an easily washable fabric. Thanks to the use of fastening straps it can be used together with **BodyMap®** cushions in order to prevent the patient from slipping out and also to stabilize the patient.

It is possible to use parts listed below together with this product:

One-sided tape with clamp TJK/63/2,5, TJK/63/3

Non-elastic stabilizing belt with clamps TONK/174/2,5 and TONK/174/3

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
6 POINTS SAFETY VEST – is a vest stabilizing the trunk. It has a wide adjustment range, which makes it easier to fit it to the user. The vest is made of an easily washable fabric and has a soft cotton inside which prevents the patient's skin from abrasions and injuries. Thanks to the use of fastening straps it can be used together with **BodyMap®** cushions in order to prevent the patient from slipping out and also to stabilize the patient. In addition, there is a possibility of quick adjustment of the vest to the patient's body with the help of handles. If necessary, you can easily unbuckle the vest.

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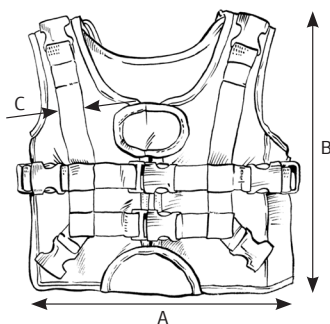
H HARNESS – is an H shaped vest stabilizing the trunk. It has a wide adjustment range, which makes it easier to fit it to the user. The vest is made of an easily washable fabric and has a soft cotton inside which prevents the patient's skin from abrasions and injuries. Thanks to the use of fastening straps it can be used together with **BodyMap**® cushions in order to prevent the patient from slipping out and also to stabilize the patient. In addition, there is a possibility of a quick adjustment of the vest to the patient's body with the help of handles. If necessary, you can easily unbuckle the vest.

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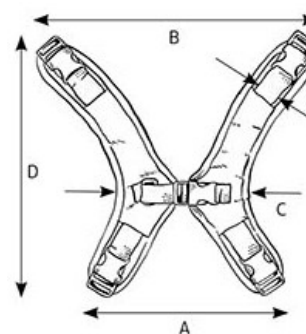It is universal and durable support with wide range of adjustments. Designed for stabilizing the upper limbs with **BodyMap® System**. Perfect for hemiplegia and hemiparesis to support and relieve upper limbs with paresis or palsy. Thanks to universal grip the support can be mounted on wheelchairs, special needs strollers, etc.

This support is designed for use with **BodyMap® E** cushion. We can fix the cushion on the support using velcro tapes available in our offer also.

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There is a possibility to order cover which is equipped with thermo-active foam that works according to two steps:

1. It adjusts to pressure which is put on patient's body. It is first, introductory level of adjustment. It is also first and last level for normal foam.
2. It is not immediate because it is spread over time. Moreover, cover softens due to patient's body heat. This effect is not the same within whole surface when body has contact with device. For this reason, foam softens in the hottest places, these "red points", where body pressure on device is still too firm. This reaction between cover and thermo-elastic foam makes it possible to distribute pressure points.

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Soft and nice to the touch cover is made of a breathable elastic fabric. It perfectly adapts to the cushion's shape. The fabric is very silken and moves moisture away from the skin. It has a special string which allows you to keep the fabric taut, and at the same time prevents the cover from moving.

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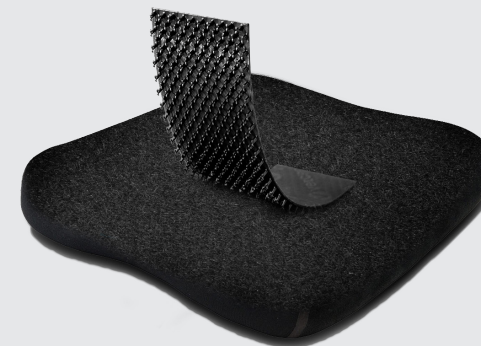
Optionally, at the customer's request, the cushion can be equipped with a self-adhesive fabric at the bottom, which makes it possible to attach various elements with the use of Velcro to it. Thanks to that function, stabilizing the patient or mounting the cushion in any place, e.g. in a stroller, chair or car seat, is now possible. Additionally, the self-adhesive function prevents the cushion from moving.

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3D MESH™ COVER is made of elastic and nice to the touch fabric. It is durable and resistant to stretching. The jersey used in the cover allows for air circulation, which is crucial in the anti-bedsores prophylaxis, at the same time protecting it against bacteria and mold.

3D MESH™ COVER is easy to clean with the use of mild cleaners. The upholstery is made of environmentally friendly fabrics.

3D MESH™ COVER provides a high comfort of use and is safe for the skin.

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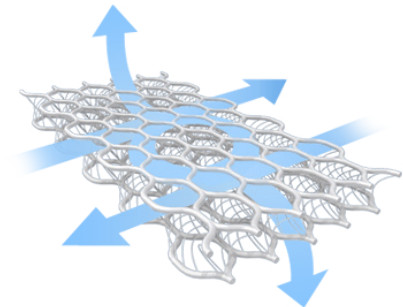
Aerating-massaging pad was created for bed sore prevention for patients that use **BodyMap®** products for a long time. This pad is used mostly in system's seats. Pad can be used only with terry cover if patient is sensitive to abrasion. Aerating-massaging pad makes it possible to lighten as well as ventilate sciatic region. Additionally, when the patient performs movement, his skin is massaged which is really beneficial due to the fact that it improves blood circulation and prevents bedsores development.

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HOW IT WORKS?
JAK TO DZIAŁA?

BODYMAP® WG GRANULES GRANULAT

As far as additional equipment is concerned, **BodyMap®** system is equipped with bags. They contain granules and they are available in two sizes. Bags are created so as to have control over the amount of granules in **BodyMap®** cushions. It is highly beneficial due to the fact that it is possible to adjust **BodyMap®** cushions to user's needs. It is necessary to take the size of **BodyMap®** cushion into account while choosing the size of bag with granules. **BodyMap®** cushion has to be equipped with service valve in order to pour in or take out granules. Service valve is situated at the back of **BodyMap®** cushion thanks to which it is not very visible and, moreover, it is not an obstacle while using the product. What is more, it is really easy to use it – you just need to unscrew cork and either pour in or take out specific amount of granules.

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The repair kit includes a tube of neoprene glue, a set of patches and instruction manual. The setenables self-repair of minor damage of the **BodyMap®** cushion. The set is added free of charge to any **BodyMap®** cushion.

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BODYMAP® Z VALVE ZAWÓR

An indispensable element of every **BodyMap®** cushion. Allows you to deflate or inflate the cushion. It is a standard equipment for every cushion.

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Optionally, the product can be equipped with a unique service valve which allows you to adjust the amount of the granules in the cushion. The valve makes it easier to increase or decrease the amount of the granules in the product depending on the individual needs. It is recommended to us the GRANULES for proper use of the valve.

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Anti-migration pad dedicated to BodyMap system pillows.

The innovative material of the pad will keep the pillow always in the right place. A special foam material with a mesh structure has excellent anti-slip properties. Effectively prevents the pillow from moving automatically when using it in a stroller, car seat or other equipment. The form and size of the pad are perfectly matched to the specific pillow model from the BodyMap range.

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The outer layer of the cover is made of elastic **PureMed™** material – its washable properties are an ideal solution while using the **BodyMap®** cushions at home, rehabilitation centers or health care facilities, where cleanliness and hygiene are of vital importance. This is especially important when more than one person is using the **BodyMap®** product.

The second layer of the cover is the **VISMemo™** memory foam which adjusts its shape to the shape of the user's body under pressure. The **VISMemo™** is most active in the areas most prone to bedsores, i.e. where the pressure is the highest.

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The stabilizing and positioning **BodyMap®** system is made of an innovative grey fabric **PureMed™** which has numerous advantages:

- It is safe for people and very easy to disinfect;
- Ideal for use in areas with the high risk of infection;
- There is a possibility of multiple use, it does not rub off;
- It is perfect when the cushions are used by different users, e.g. in hospitals or rehabilitation centers;
- It is lightweight, soft, elastic and easy to shape;
- Contains antibacterial components based on inorganic Silver Zeolite;
- It meets all the essential requirements of the medical products of different kinds*;
- It has **OEKO-TEX® STANDARD 100** certification which is granted to exceptionally skin-friendly parameters;
- It is used as a cover for hospital mattresses, operating tables and couches;
- It is waterproof and vapour-permeable in one way only which means that the air from the inside of the cushion may migrate freely outside through the **PureMed™** cover.
- At the same time, liquids such as sweat, blood or urine will not be able to get through the cover inside the cushion;
- The **PureMed™** cover has passed the **CIGARETTE TEST** and the **MATCHSTICK TEST** positively, which proves that it is a non-flammable fabric;
- Complies with **REACH** regulation aimed at ensuring a high level of health and environmental protection.

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WATERPROOF PRODUCT

The layer of the cover is made of elastic **PureMed™** material – its washable properties are an ideal solution while using the **BodyMap®** cushions at home, rehabilitation centers or health care facilities, where cleanliness and hygiene are of vital importance. This is especially important when more than one person is using the **BodyMap®** product.

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WATERPROOF PRODUCT

A lap tray is a multifunctional and portable tray. A special bag filled with granules makes it possible to stably place it on your knees or on other uneven surfaces. Perfect for home or school, during therapy activities, everyday activities and recreation activities. An ergonomic top made of a non-slip material has a special edge which prevents objects from slipping from the tray. There is a bag filled with granules attached to the bottom with Velcro. The bag has a special pocket which allows you to adjust the amount of the granules. The tray can be used with **BodyMap® R/R+**. The tray is available in five sizes.

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