

KOD KATALOGOWY	OKD-40
NAZWA HANDLOWA	ORTEZA KOŃCZYNY DOLNEJ
MODEL	OKD-40
CODE	OKD-40
TRADE NAME	LOWER-EXTREMITY SUPPORT
MODEL	OKD-40



- 1. PL:** Przed założeniem ortozy, rozepnij wszystkie pasy. Załóż ortezę na nogę i podciągnij do góry w taki sposób, aby rzepka znalazła się w otworze okolozępkowym (1), a zawiasy szyny w osi stawu kolanowego. Tylko prawidłowo założona orteza pozostaje w osi stawu kolanowego i przylega równomiernie do całej kończyny.

EN: Before putting on the brace, unfasten all the straps. Put the brace on the leg and pull it upward so that the patella is positioned in the patellar opening (1), and the stabilizers align with the axis of the knee joint. Only a properly fitted brace stays aligned with the axis of the knee joint and fits evenly along the entire limb.



- 2. PL:** Przepleć taśmy przez klamki i zapnij na rzepy w podanej kolejności (2,3,4,5).

EN: Interlace the straps through the buckles and fasten the Velcro straps according to the following steps (2,3,4,5).



- 3. PL:** Gotowy wyrób.
EN: The product is ready to use.